


Minivette® POCT

Collection and dispensing of small blood samples for Point-of-Care tests



Minivette® POCT

- Easy sample recovery due to optimal capillary features
- Precise dispensing of small whole blood volumes
- Prevents spillage during transfer to test card
- Available in 10 µl, 20 µl, 50 µl, 100 µl and 200 µl volumes
- Preparations: Neutral, Heparin and EDTA

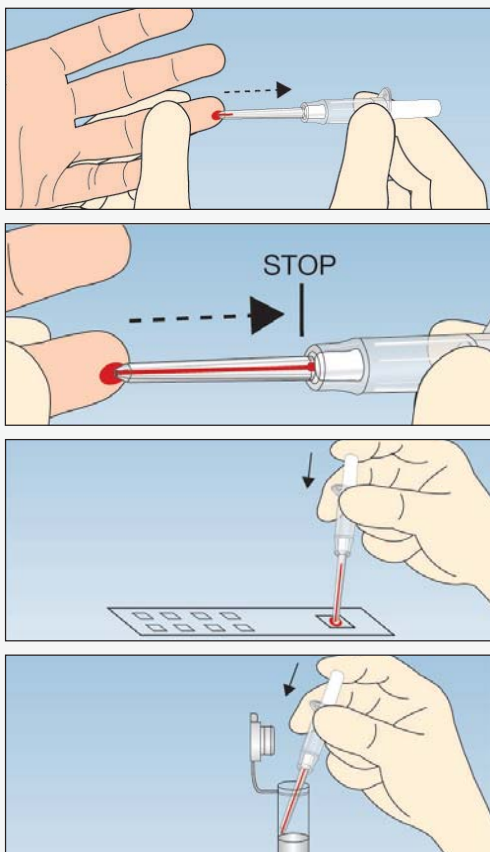

1 Hold the Minivette® POCT in a horizontal, slightly declined position, under the wings. The air ventilation hole at the end of the piston should not be covered, nor should the piston be pushed down. Collect the blood drops using the capillary tip ensuring there are no air bubbles present.

2 Blood collection is complete when the blood has reached the filter at the other end of the capillary tip.

3a Place the capillary tip on the test field and slightly press down the piston to dispense the whole blood sample.

3b Alternatively, dispense the sample into a micro tube.

All patient blood samples must be treated as potentially infectious. Wear gloves!

Ordering Information

Order No.	Description	Color Code	Packaging
NEW 17.2111.920	Minivette® POCT with cap	white	ind. wrapped 100 pcs./inner box / 1.000 pcs./case
17.2111.010	Minivette® POCT 10 µl neutral	white	200 pcs./inner box / 2.000 pcs./case
17.2111.020	Minivette® POCT 20 µl neutral	white	
17.2111.050	Minivette® POCT 50 µl neutral	white	
17.2111.100	Minivette® POCT 100 µl neutral	white	
17.2111.200	Minivette® POCT 200 µl neutral	white	
17.2112.020	Minivette® POCT 20 µl Lithium Heparin	orange	200 pcs./inner box / 2.000 pcs./case
17.2112.120	Minivette® POCT 20 µl Lithium Heparin	green	
17.2112.050	Minivette® POCT 50 µl Lithium Heparin	orange	
17.2112.150	Minivette® POCT 50 µl Lithium Heparin	green	
17.2112.100	Minivette® POCT 100 µl Lithium Heparin	orange	
17.2112.101	Minivette® POCT 100 µl Lithium Heparin	green	
17.2112.200	Minivette® POCT 200 µl Lithium Heparin	orange	150 pcs./inner box / 1.500 pcs./case
17.2112.201	Minivette® POCT 200 µl Lithium Heparin	green	
17.2113.020	Minivette® POCT 20 µl EDTA	red	200 pcs./inner box / 2.000 pcs./case
17.2113.120	Minivette® POCT 20 µl EDTA	violet	
17.2113.050	Minivette® POCT 50 µl EDTA	red	
17.2113.150	Minivette® POCT 50 µl EDTA	violet	
17.2113.101	Minivette® POCT 100 µl EDTA	violet	
17.2113.201	Minivette® POCT 200 µl EDTA	violet	150 pcs./inner box / 1.500 pcs./case

SARSTEDT AG & Co.
P.O. Box 12 20
D-51582 Nümbrecht
Phone +49 2293 3050
Fax +49 2293 305-122
info@sarstedt.com
www.sarstedt.com