



# Wichtige Handhabungshinweise

## Blutgas-Monovette®

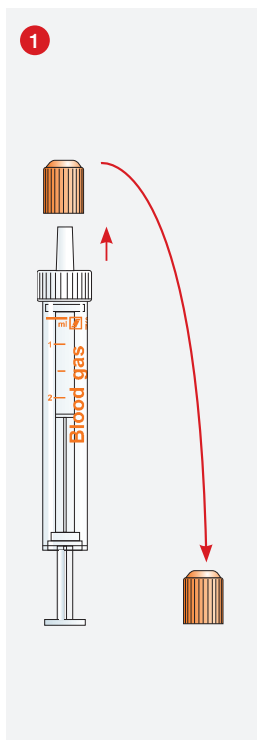


Entfernen Sie die orangefarbene Schutzkappe von der Blutgas-Monovette®.

### Blutentnahme mit Safety-Kanüle oder Safety-Multifly®-Kanüle

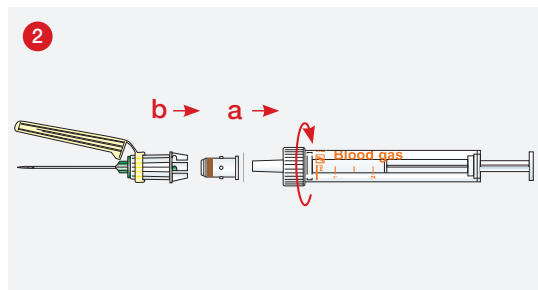


Setzen Sie den Membran-Adapter (Art. Nr.: 14.1112) auf den Luer-Konus der Blutgas-Monovette® (a) und komplettieren Sie die Blutgas-Monovette® mit der Safety-Kanüle (b) oder der Safety-Multifly®-Kanüle.

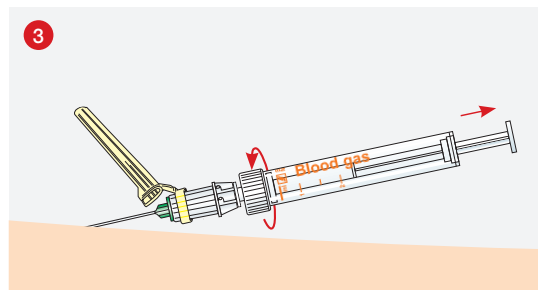


Entnehmen Sie die Blutprobe entsprechend Ihrer Arbeitsanweisung.

### Blutentnahme über Gefäßzugang

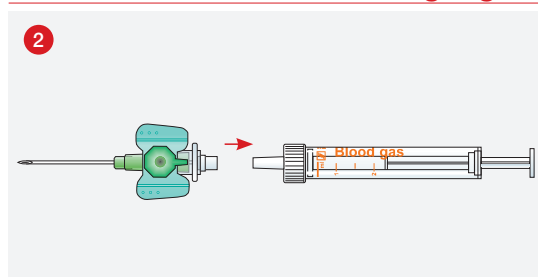


Verbinden Sie die Blutgas-Monovette® mit dem Gefäßzugang.

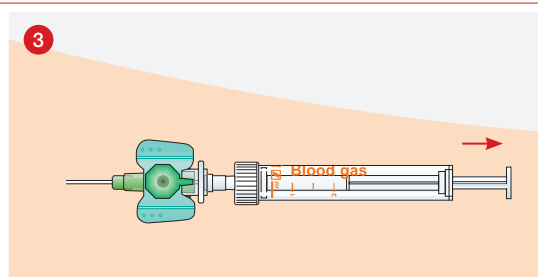


Entnehmen Sie die Blutprobe durch langsames Zurückziehen der Kolbenstange entsprechend Ihrer Arbeitsanweisung.

## Probenhandling

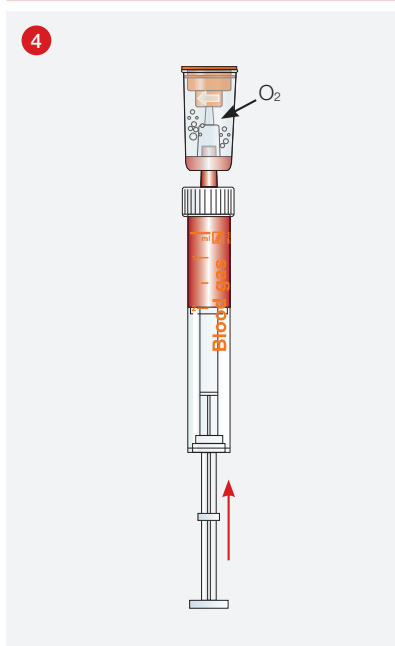


### WICHTIG!

Zur Vermeidung von Fehlmessungen aufgrund von Luftkontamination, entfernen Sie die Luft nach Aufsetzen des Entlüfters (Art. Nr.: 14.1148) durch vorsichtiges Hochdrücken des Kolbens.

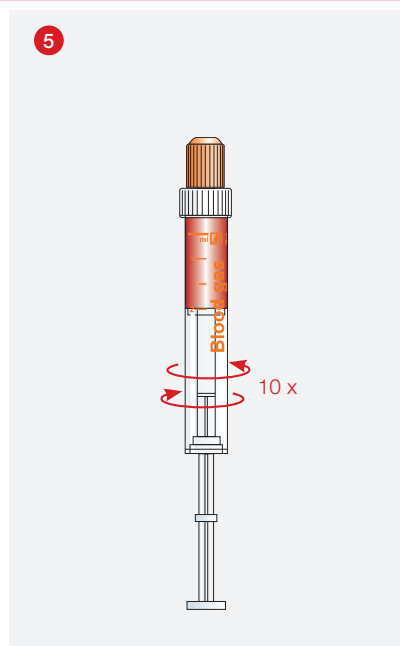


Mischen Sie die Blutprobe unmittelbar nach der Entnahme indem Sie die Blutgas-Monovette® zwischen den Handflächen rollen.

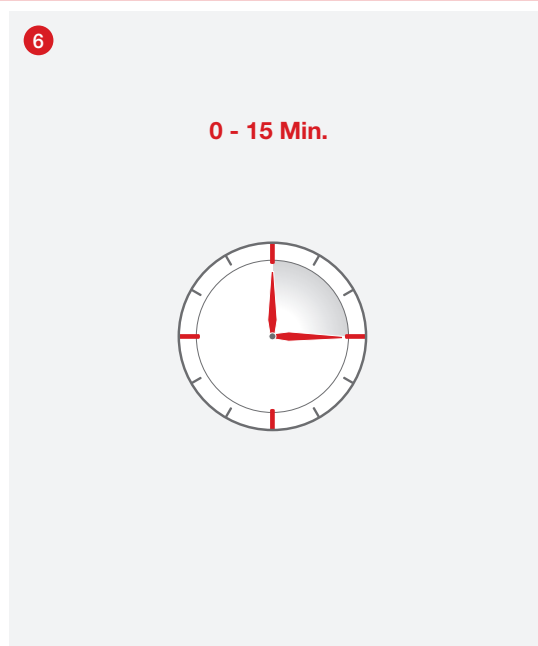

Blutgasanalysen sollten möglichst sofort nach der Blutentnahme, spätestens 15 Minuten nach der Blutentnahme durchgeführt werden.


# Important Instructions for Use

## Blood Gas Monovette®

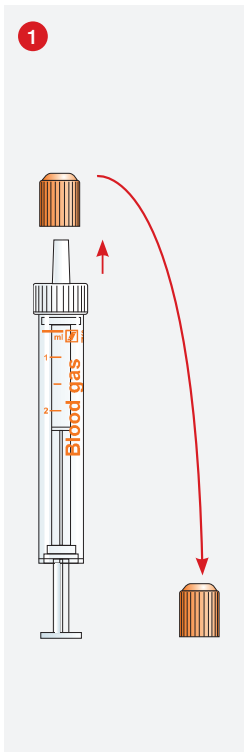


Remove the orange protective cap from the Blood Gas Monovette®.

### Blood collection with the Safety-Needle or Safety-Multifly-Needle®

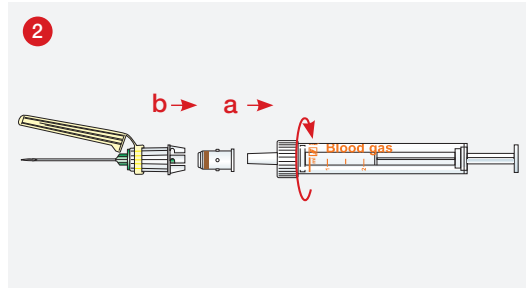


Attach the membrane-adaptor (Art. No. 14.1112) to the Luer cone of the Blood Gas Monovette® (a) and connect the Blood Gas Monovette® with the Safety-Needle (b) or the Safety-Multifly® Needle.

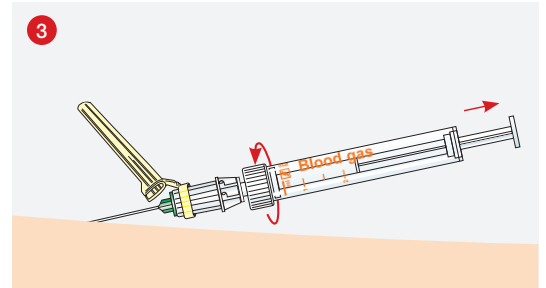


Collect blood in accordance with your standard procedures.

### Blood collection via an indwelling cannula

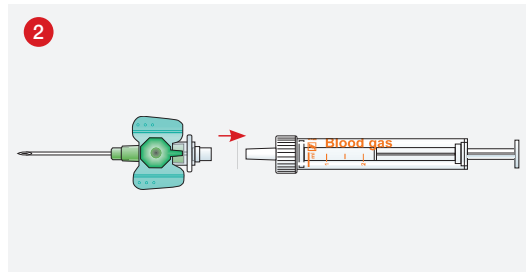


Connect the Blood Gas Monovette® with the indwelling cannula.

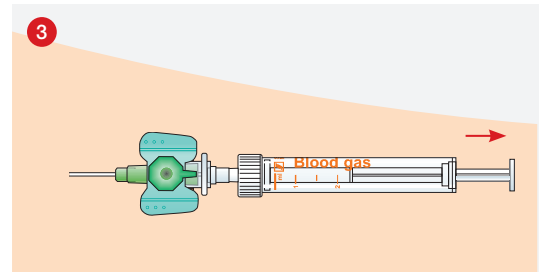


Collect blood by slowly withdrawing the piston in accordance with your standard procedures.

### Sample handling

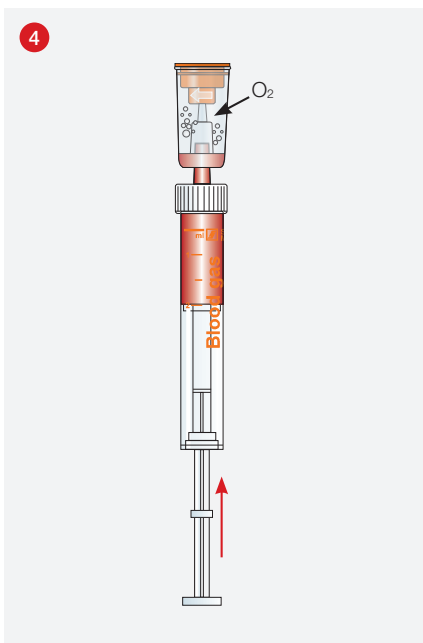


#### IMPORTANT!

To prevent false results caused by contamination with air, remove the air by attaching the ventilation device (Art. No. 14.1148) and carefully pushing the piston upwards.

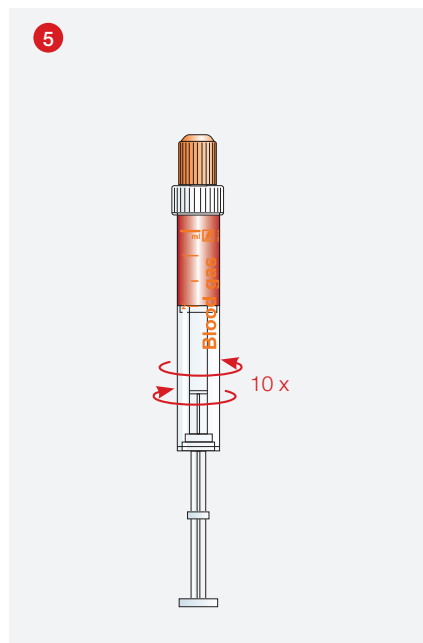

Mix the blood sample immediately after collection by rolling the Blood Gas Monovette® between your palms.

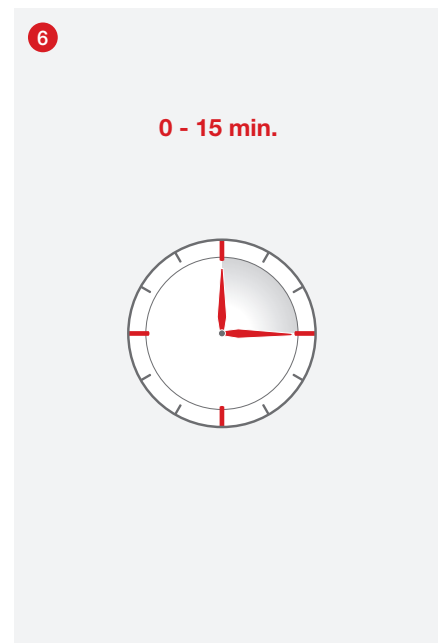

For best results, blood gas analyses should be carried out immediately or no later than 15 minutes after blood collection.