

# NFT

## Urine collection system

Needle-free, hygienic,  
enclosed collection



# NFT

# Needle Free Transfer

Urine  
pre-analytics  
2.0

The first urine collection system from SARSTEDT for hygienic and needle-free sample collection

## Hygienic. Practical. Safe.

How can collecting urine samples be made even safer, easier and more hygienic? The answer to this question can be found in the innovative products of the NFT system. As the first enclosed and needle-free urine collection system from SARSTEDT, the new product line combines the advantages of existing products in a single solution.


## New standards for urine pre-analytics



- Available as an NFT urine cup, as well as an NFT 3-liter urine container for 24 hour urine collection
- Needle-free collection system
- Safe storage and transport
- Easy, hygienic urine transfer with the Urine-Monovette®
- No sharps container needed to dispose of the NFT products

### Simply practical

The Urine-Monovette® enables hygienic urine collection and is also suitable as a primary tube for test strips and analyzers and as a centrifuge tube for collecting sediment

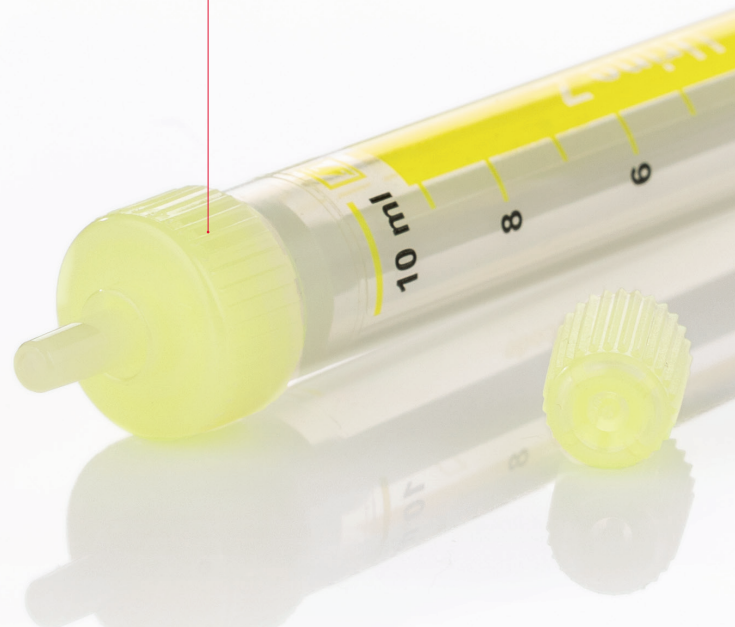


## New

A needle is no longer needed with the new needle-free collection function. The Urine-Monovette® easily pierces the innovative NFT membrane.


Following an easy collection, the opening is immediately closed and hygienically sealed even after collecting several samples.

There is no risk of infection from a needlestick injury.

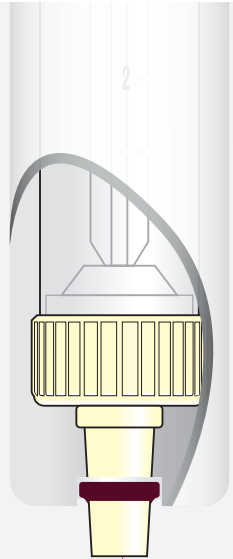

# CONVINCING

- ✓ **Minimized risk of injury** and **increased user safety** for personnel and patient through needle-free sample collection
- ✓ **Savings on disposal**, as the components of the NFT system do not need to be disposed of separately in a sharps container

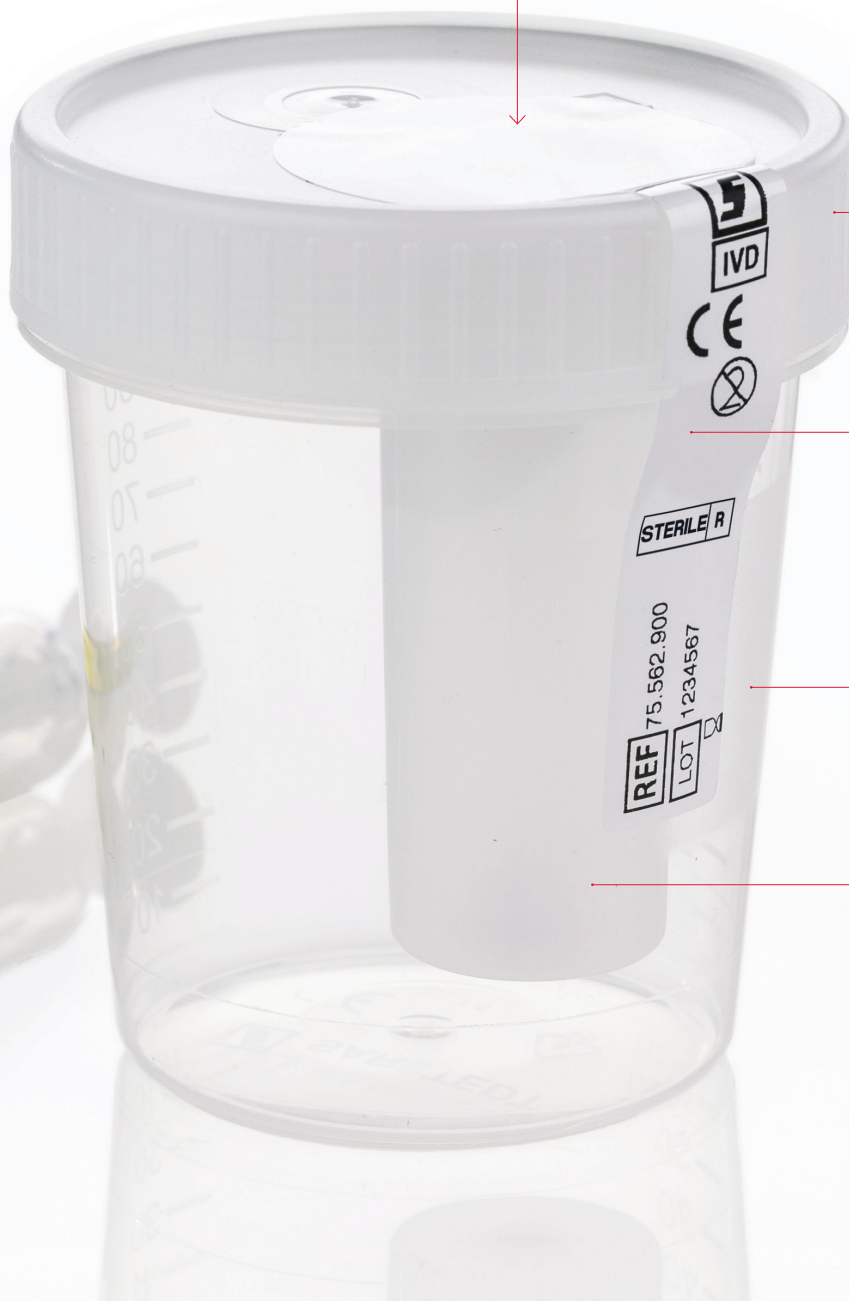

**Liquid and odor tight**  
Screw cap with transfer unit

**Sealed**  
If intact, the safety label indicates the internal sterility of the NFT urine cup

**Clear**  
Highly transparent container of robust polypropylene (PP) for good visual assessment

**Innovative**  
Needle-free transfer unit with pierceable NFT membrane

# NFT

## Urine collection system

The best from various systems

The NFT system with Urine-Monovette®, NFT urine cup and NFT urine container ensures hygienic working conditions during the entire sample collection process:

- With the Urine-Monovette® and the enclosed suction tip, you can carry out the transfer process without a needle, even if the filling volume is very small. A separate transfer device as in vacuum systems is not needed.
- Extensive training is not required.
- With the Urine-Monovette®, the aspiration principle allows the collection process to be repeated if necessary – there is no risk of loss of function through repeated use.
- The well-designed, needle-free collection function minimizes the risk of injury for staff and patient.
- After collection, the products of the NFT system do not need to be disposed of separately in needle-proof containers – saving time and money compared with products with an integrated needle.

### Well designed

The NFT urine container with a volume of 3,000 ml, ergonomic grip, large opening diameter and transparent inspection window with graduated scale in increments of 100 ml



## Urine-Monovette® – hygienic work with just a few movements

- The Urine-Monovette® enables hygienic, needle-free sample collection from urine containers or urine drainage systems.
- A urine analysis with commercially available test strips can be carried out directly in the filled Urine-Monovette®.
- The Urine-Monovette® is suitable for transporting and shipping according to the P650 packaging instructions of the ADR and IATA. It can be used in centrifuges and analyzers for collecting sediment.



## User-friendly handling: NFT urine cup



## Convenient for 24-hour urine collection: NFT urine container



The large-volume NFT urine container also ensures sample collection remains hygienic thanks to the enclosed collection function.



Watch the tutorial on the easy handling of the NFT products: [sarstedt.com/nft-us](https://sarstedt.com/nft-us)

# ORDER A FREE SAMPLE



[sarstedt.com/nft-sa-us](http://sarstedt.com/nft-sa-us)



## SARSTEDT sets new standards with the NFT urine collection system

Only clean and precise sample collection can ensure correct analysis results. The NFT system from SARSTEDT sets a high standard for accurate pre-analytics, it is suitable for routine use, and provides the greatest safety for user and patient.



NFT products do not need to be disposed of in sharps containers.


## Pre-analytical workflow powered by SARSTEDT

Take advantage of the synergy of our synchronized systems.

Discover the 360° solutions for your pre-analytics from SARSTEDT



[sarstedt.com/nft-wf-us](http://sarstedt.com/nft-wf-us)

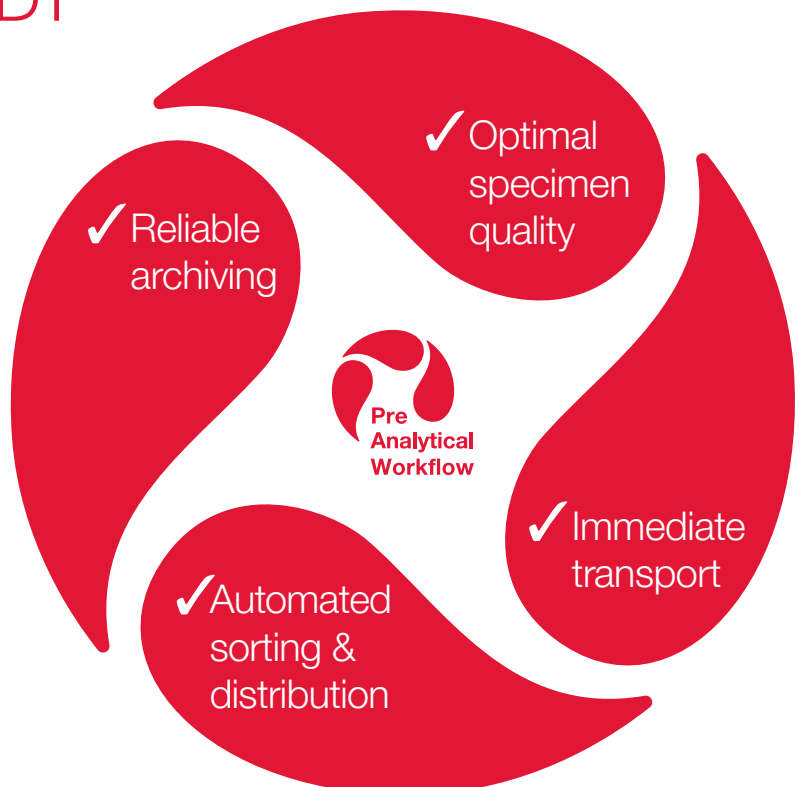

**SARSTEDT**

## NFT urine cup 100 ml

Order no.	Volume	L x Ø	Version	Purity	Packaging (inner/outer box)
75.562.900	100 ml	72 x 62 mm	With safety label and transfer unit	Sterile	200 / 200

## NFT urine container 3.0 l

Order no.	Volume	L x W x H	Version	Purity	Packaging (inner/outer box)
77.575.900	3000 ml	125 x 130 x 246 mm	With enclosed transfer unit and patient label, opening diameter 80 mm	–	30 / 30

## Urine-Monovette® Z - without additive

Order no.	Volume	L x Ø	Version (yellow)	Purity	Packaging (inner/outer box)
10.250.001	3.2 ml	75 x 13 mm	Paper label, writable	–	64 / 512
10.258	8.5 ml	92 x 15 mm	Plastic label, transparent	–	64 / 512
10.258.001	8.5 ml	92 x 15 mm	Paper label, writable	–	64 / 512
10.258.020	8.5 ml	92 x 15 mm	Plastic label, transparent, individually blister packed	Sterile	100 / 500
10.252	10 ml	102 x 15 mm	Plastic label, transparent	–	64 / 512
10.252.001	10 ml	102 x 15 mm	Paper label, writable	–	64 / 512
10.252.020	10 ml	102 x 15 mm	Plastic label, transparent, individually blister packed	Sterile	100 / 500
10.252.028	10 ml	102 x 15 mm	Plastic label, transparent, individually blister packed	Sterile	100 / 500
10.252.030	10 ml	102 x 15 mm	Plastic label, transparent, with light protection	–	64 / 512

## Urine-Monovette® Boric acid - with stabilizer for microbiological tests

Order no.	Volume	L x Ø	Version (green)	Purity	Packaging (inner/outer box)
10.256.001	3.2 ml	75 x 13 mm	Paper label, writable	–	64 / 512
10.256.021	3.2 ml	75 x 13 mm	Paper label, writable, individually blister packed	Sterile	100 / 500
10.260.001*	8.5 ml	92 x 15 mm	Paper label, writable	–	64 / 512
10.260.021*	8.5 ml	92 x 15 mm	Paper label, writable, individually blister packed	Sterile	100 / 500
10.253	10 ml	102 x 15 mm	Plastic label, transparent	–	64 / 512
10.253.001	10 ml	102 x 15 mm	Paper label, writable	–	64 / 512
10.253.020	10 ml	102 x 15 mm	Plastic label, transparent, individually blister packed	Sterile	100 / 500
10.253.021	10 ml	102 x 15 mm	Paper label, writable, individually blister packed	Sterile	100 / 500

For more consumables for urine pre-analytics, see [www.sarstedt.com](http://www.sarstedt.com)

\*Planned introduction in the first quarter of 2024

## SARSTEDT, Inc.

1025 St. James Church Road

P.O. Box 468

Newton, NC 28658 - 0468

Toll free: +1 800 257 5101

Fax: +1 828 465 4003

[customerservice.us@sarstedt.com](mailto:customerservice.us@sarstedt.com)

[www.sarstedt.com](http://www.sarstedt.com)



Further information,  
videos and sample order:  
[sarstedt.com/nft-us](http://sarstedt.com/nft-us)

If you have any questions:  
We'd be happy to help!

Visit our website: [www.sarstedt.com](http://www.sarstedt.com)