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Produktspezifische Normen:

DIN EN 14254: *In-vitro*-diagnostic medical devices - Single-use receptacles for the collection of specimens, other than blood, from humans

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Intended use

The Monovette® VD specimen collection container is used to collect saliva samples in a gargle solution and aids in the collecting, transport, processing and storage of saliva samples in a gargle solution for *in vitro* diagnostics (e.g. SARS CoV-2 detection and other virus diagnostics).

The VD specimen collection screw top cup is used to collect saliva samples in a gargle solution and aids in collection with the Monovette® VD.

The products are available individually and can be used together.

The products are intended for use in a professional environment by qualified medical and laboratory personnel.

Product description

The Monovette® VD and VD screw top cup consist of plastic containers.

The Monovette® VD has a light blue Luer cap, a light blue cap and an enclosed suction tip.

The VD screw top cup has a light blue screw cap.

The products do not contain any preparations.

Name	Dimensions
Monovette® VD 10 ml	10 ml 102 x 15 mm
Monovette® VD 8.5 ml	8.5 ml 92 x 15 mm
VD screw top cup	max. 100 ml 73 x 62 mm

Safety information and warnings

1. General precautions: Use gloves and general personal protective equipment to protect yourself from possible exposure to potentially infectious specimen material and transmitted pathogens.
2. Handle all biological specimens and collection utensils according to the guidelines and procedures in your facility. In the event of direct contact with biological samples, consult a doctor, as they can transmit viral diseases or other infectious diseases. Always follow the safety guidelines and procedures of your facility.
3. Dispose of all collection utensils in containers for hazardous biological waste.
4. Do not use the products after the expiry date. The expiry date of the product is the last day of the month and year as indicated.

Storage

Store the products at room temperature.

Transport

Monovette® VD: The products can be transported together with an ADR-compliant (Packing instruction P650) secondary vessel.

Limitations

The storage period and temperature for a filled Monovette® VD depend on the expiry date of the parameters for investigation. Therefore, and because of the onset of bacterial growth in saliva after a few hours at room temperature, it is recommended to analyse the firmly closed Monovette® VD immediately after collecting the specimen or to store it in the refrigerator until further processing.

These products may not be used for children under the age of 3 or for persons with a higher risk of choking.

Collecting and handling specimens










READ THIS DOCUMENT COMPLETELY BEFORE STARTING A GARGLE SOLUTION TRANSFER.

Material needed for the gargle solution transfer:

1. Gloves, gown, goggles or other suitable protective wear to protect against potentially infectious materials.
2. Required number of Monovettes VD and VD screw top cups
3. Disposal container for hazardous biological material for the safe disposal of used materials.

Intended use:

- 1 Note: Before collecting saliva, the patient should not eat, drink, brush their teeth, take medication or smoke for at least 60 min before the test.
- 2 Unscrew the cup and give the cup to the patient without the lid.
- 3 The patient gargles with a gargle solution selected by the doctor.
- 4 The patient spits the gargle solution into the cup (follow laboratory directions).
- 5 Take the cap off the Monovette® VD and set it aside. Attach the suction tip.
- 6 Take the cup with the saliva sample and dip the suction tip into the sample material. Draw the Monovette® VD up to the stop mark.
- 7 Rotate the Monovette® VD 180°.
- 8 Hold the Monovette® VD upright and pull the plunger rod downwards to the stop to completely empty the suction tip.
- 9 Take the suction tip off and snap off the plunger rod. Put the cap back on the Monovette® VD.
- 10 Dispose of the suction tip, the plunger rod and the cup in a disposal container for hazardous biological waste.

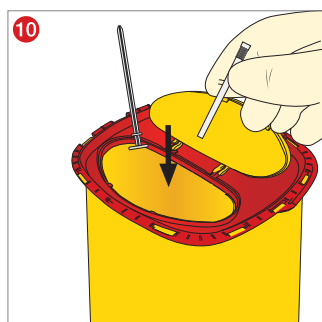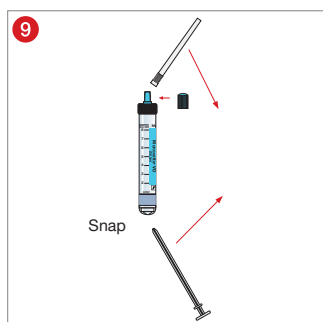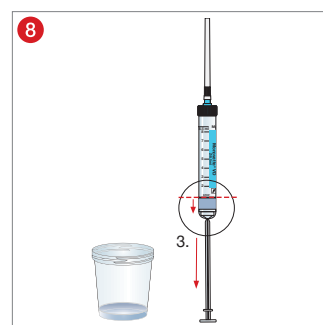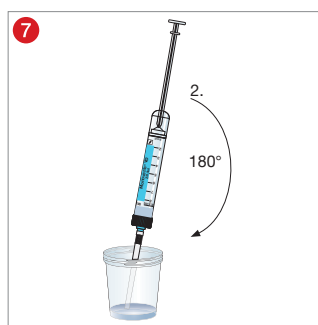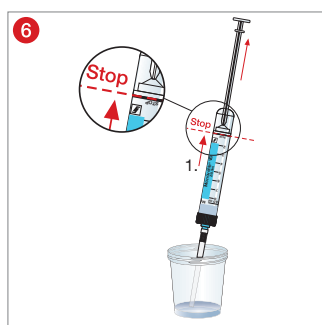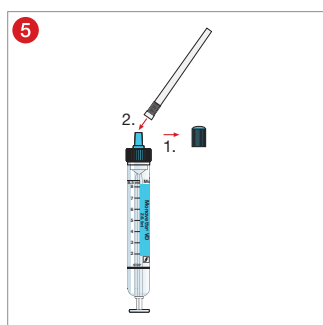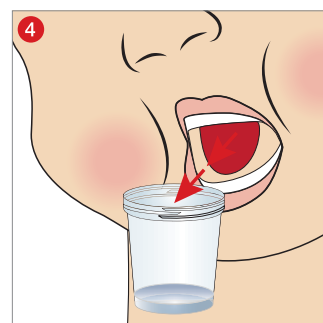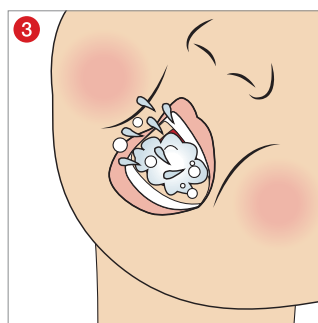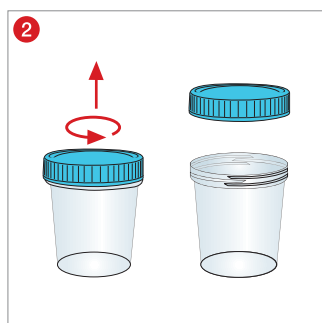Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. Contaminated or filled Monovettes VD and VD screw top cups must be disposed of in suitable containers for hazardous biological waste.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).

ISO standards (as amended/updated)

Product-specific standards:

DIN EN 14254: *In vitro* diagnostic medical devices – Single-use receptacles for the collection of specimens, other than blood, from humans

Key for symbols and labels:

	Article number
	Batch number
	CE marking
	<i>In vitro</i> diagnostic device
	Follow the instructions for use
	If reused: Risk of contamination
	Manufacturer
	Country of manufacture

Technical modifications reserved.

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

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