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## SARSTEDT Micro-Kanüle

<b>DE</b>	Gebrauchshinweis – SARSTEDT Micro-Kanüle	1 - 4
<b>EN</b>	Instructions for Use – SARSTEDT Micro Needle	5 - 8
<b>BG</b>	Инструкции за употреба – SARSTEDT микроканюла	9 - 12
<b>CS</b>	Návod k obsluze – Mikro Jehla SARSTEDT	13 - 16
<b>DA</b>	Bruksanvisning – SARSTEDT Mikrokanyl	17 - 20
<b>EL</b>	Οδηγίες χρήσεως – Μικροβελόνα SARSTEDT	21 - 24
<b>ES</b>	Instrucciones de uso – Micro-aguja SARSTEDT	25 - 28
<b>ET</b>	Kasutusjuhend – SARSTEDT mikrokanüül	29 - 32
<b>FR</b>	Mode d'emploi – Micro-aiguille SARSTEDT	33 - 36
<b>HR</b>	Uputa za upotrebu – SARSTEDT Mikro-igla	37 - 40
<b>HU</b>	Használati utasítás – SARSTEDT mikrokanülök	41 - 44
<b>IT</b>	Istruzioni d'uso – Micro-ago SARSTEDT	45 - 48
<b>KO</b>	사용 설명서 – SARSTEDT 마이크로 니들 (바늘)	49 - 52
<b>LT</b>	Naudojimo instrukcija – SARSTEDT mikrokaniulė	53 - 56
<b>LV</b>	Lietošanas norādes – SARSTEDT mikroadata	57 - 60
<b>NL</b>	Gebruiksaanwijzing – SARSTEDT-microcanule	61 - 64
<b>NO</b>	Bruksanvisning – SARSTEDT Micro-kanyle	65 - 68
<b>PL</b>	Instrukcja obsługi – Mikroigła SARSTEDT	69 - 72
<b>PT</b>	Instruções de utilização – Micro agulha SARSTEDT	73 - 76
<b>RO</b>	Instrucțiuni de utilizare – Microcanula SARSTEDT	77 - 80
<b>RU</b>	Инструкция по применению – Микроканюля SARSTEDT	81 - 84
<b>SK</b>	Návod na Použitie – SARSTEDT Mikroihla	85 - 88
<b>SL</b>	Navodila za uporabo – Mikrokanila SARSTEDT	89 - 92
<b>SV</b>	Bruksanvisning – SARSTEDT mikrokanylen	93 - 96
<b>TH</b>	คำแนะนำในการใช้งาน – หัวเข็มขนาดเล็กของ SARSTEDT	97 - 100
<b>TR</b>	Kullanım için talimatlar – SARSTEDT Mikro iğne	101 - 104
<b>ZH</b>	使用说明 – SARSTEDT 微型采血器	105 - 108

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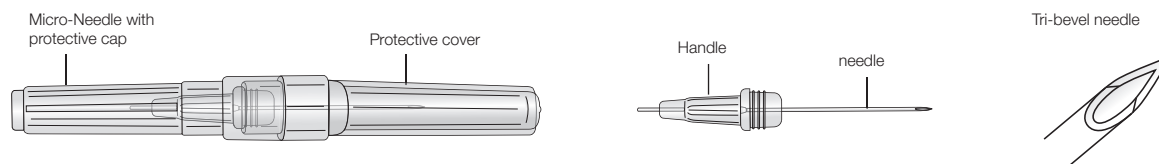
## Intended use

The Micro-Needle is a sterile, single-use needle developed for drawing small volumes of blood. The Micro-Needle, with a prepared sample tube, e.g. micro sample tube, is a system for drawing venous blood. The products are to be used by trained personnel.

## Product description

The Micro-Needle is a sterile, pyrogen free, disposable product packaged in individual blisters. The Micro-Needle consists of a 3/4" (19 mm) long disposable, tri-bevel, stainless steel medical needle with a handle, a protective cap and a protective sleeve. The products are available in the needle diameters 21, 23 and 25 gauge (G).

### Micro-Needle

## Safety information and warnings

1. Check all individual packaging for damage prior to use and, if damaged, do not use. Likewise do not use the product if the needle is bent or broken.
2. Follow the directions for handling.
3. Wear gloves and general personal protective equipment to protect yourself from blood and possible exposure to pathogens transmitted by blood.
4. Do not bend the needle.
5. During use and disposal, always keep your hands behind the needle.
6. When removing or disposing of the Micro-Needle, watch out for blood that may drip out. It is recommended that you hold the Micro-Needle in a horizontal position if possible when removing and disposing of it.
7. Dispose of used, contaminated Micro-Needles immediately in suitable sharps disposal containers and dispose of blood collection equipment (e.g. lancets, adapters) in sharps disposal containers as well.
8. Treat all biological specimens and sharp blood collection utensils (needles) according to the guidelines and procedures in your institution. In case of exposure to biological specimens or a needle prick injury, consult a doctor, as HBV, HCV, or HIV or other infectious diseases can be transmitted. Use needles with an integrated needle protector. The safety guidelines and procedures of your institution must always be followed.
9. Reusing this product could result in infections, injury or death.

## Contraindications

1. Do not perform venous blood collection on patients with infections, burns, injuries or after vascular surgery at the planned puncture site.
2. Do not perform injections of any kind with the Micro-Needle; it is to be used only for blood collection.

## General residual risks of venous blood collection

When applying the basic technique for blood collection, there are the following general residual risks according to CLSI GP41 and the WHO guidelines on drawing blood:

1. Pain
2. Bruising
3. Vasovagal reactions
4. Infections
5. Nerve injuries
6. Allergic reactions, mainly caused by disinfectants.

## Collecting and handling specimens

**READ THIS DOCUMENT IN FULL BEFORE PROCEEDING WITH VENIPUNCTURE.**

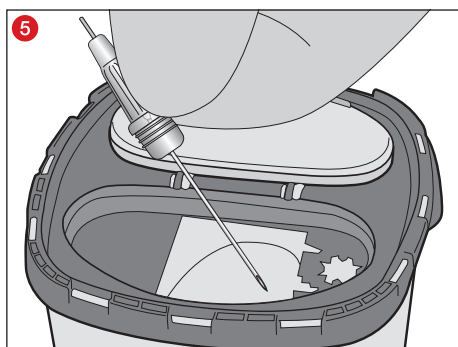
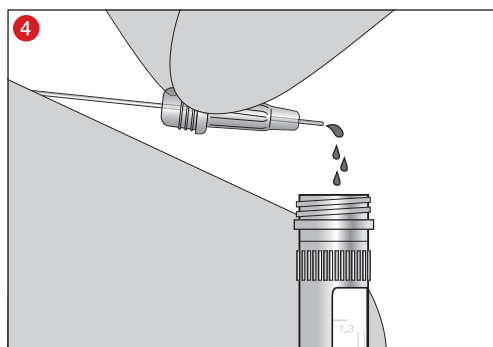
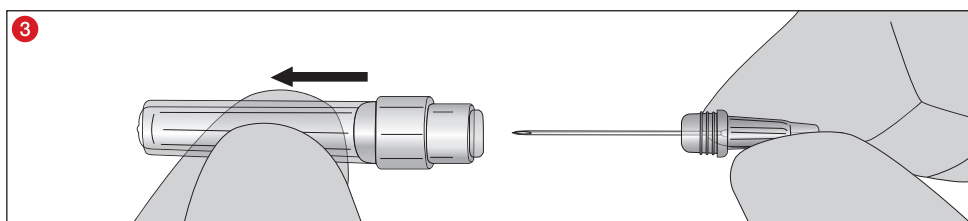
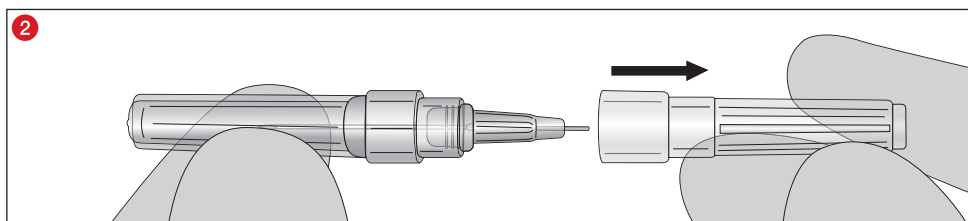
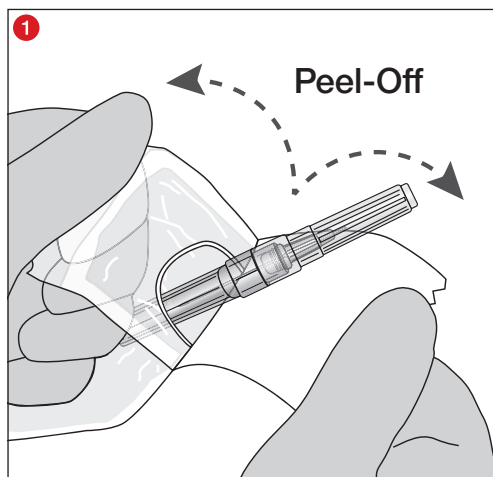
### EQUIPMENT REQUIRED FOR SPECIMEN COLLECTION

1. All necessary sample containers, e.g. micro sample tubes.
2. Micro-Needle.
3. Gloves, gown, goggles or other suitable protective gear to protect against pathogens or potentially infectious materials transmitted by blood.
4. Labels for patient identification.
5. Disinfectant to clean the collection site (follow the guidelines of the facility for sterile specimen collection for preparing the collection site). Do not use any alcohol-based cleaning materials if the specimens are to be used for a blood alcohol test.
6. Dry, clean, disposable swabs.
7. Tourniquet if needed.
8. Plaster or bandage.
9. Sharps disposal container for the safe disposal of used material.

## Venipuncture with the Micro-Needle

Apply a tourniquet for drawing blood from the arm vein. Prepare the puncture site with a suitable disinfectant. Do not touch the venipuncture site after disinfection. Position the patient such that the vein is easy to access.

- 1 The blister package is opened aseptically using the peel-off technique with the thumb and index finger. The transparent film must be facing upwards.
- 2 Remove the protective cap of the Micro-Needle.
- 3 After the disinfectant has dried, remove the protective cover of the Micro-Needle and ensure that the bevelled end of the Micro-Needle is pointing upwards.
- 4 Puncture the vein with the Micro-Needle by holding the Micro-Needle at the grip handle with the thumb and index finger. Hold the Micro-Needle so that the back end is below the puncture site. Allow blood to drip into the sample tubes one after another. If the blood flow slows down, the Micro-Needle can be safely rotated up to 360° using the grip handle.
- 5 After completing blood collection, the Micro-Needle is withdrawn from the vein and immediately disposed of in a sharps container.

## Storage

Store the Micro-Needle at room temperature.

NOTE: Do not expose to direct sunlight. Exceeding the maximum recommended storage temperature can impair the quality of the Micro-Needle.

## Product-specific standards and guidelines in the respective valid version

CLSI\* GP41 Procedures for the Collection of Diagnostic Blood Specimens by Venipuncture.

\*CLSI (Clinical and Laboratory Standards Institute)

WHO guidelines on drawing blood: best practice in phlebotomy

DIN 13097-4: Hypodermic needles - Part 4: Point geometry, requirements and testing.

DIN EN ISO 6009: Hypodermic needles for single use - Colour coding for identification.

DIN EN 10088-1: Stainless steels - Part 1: List of stainless steels.

DIN EN ISO 9626: Stainless steel needle tubing for the manufacture of medical devices - Requirements and test methods.

## Key for symbols and labels:

	Medical device		Use by
	Order no.		Follow the instructions for use
	Batch number		Country of manufacture
	CE mark		Keep away from sunlight
	Manufacturer		Store in a dry place
	Store at room temperature		Do not use if package is damaged
	Sterilisation by irradiation		non-pyrogenic
	Risk of contamination if reused		Single sterile barrier system
	Do not resterilise		Single sterile barrier system with protective packaging outside

Technical modifications reserved.

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

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