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SARSTEDT Kanülenhalter

DE	Gebrauchshinweis – SARSTEDT Kanülenhalter	2 - 4
EN	Instructions for Use – SARSTEDT Needle holder	5 - 7
ET	Kasutusjuhend – SARSTEDT-i kanüülihoidik	8 - 10
FR	Mode d'emploi – Porte-aiguille SARSTEDT	11 - 13
LT	Naudojimo instrukcijos – SARSTEDT kaniulių laikiklis	14 - 16
NL	Gebruiksaanwijzing – SARSTEDT-naaldhouder	17 - 19
PL	Instrukcja obsługi – Uchwyt do igły SARSTEDT	20 - 22
RU	Инструкция по применению – Держатель SARSTEDT	23 - 25
TR	Kullanım için talimatlar – SARSTEDT kanül tutucular	26 - 28
ZH	使用说明 – SARSTEDT 针管支架	29 - 31

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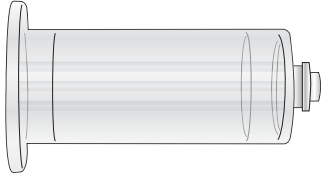
Intended use

The needle holder, together with a blood culture adapter (Art. no. 14.1110), a vacuum tube and an S-Monovette® (safety) needle or (Safety) Multify® needle is a system for venous blood collection. The product is intended for use in a professional environment by qualified medical and laboratory personnel.

Product description

The needle holder consists of plastic and was developed for connection to an S-Monovette® (safety) needle or (Safety) Multify® needle. The needle holder is a disposable product.

Safety information and warnings

1. General precautions: Use gloves and other general personal protective equipment to protect yourself from blood and possible exposure to potentially infectious specimen material and transmissible pathogens.
2. Handle all biological samples and sharp blood collecting utensils (needles) in accordance with the guidelines and procedures in your facility. In case of direct contact with biological specimens or of a needlestick injury, consult a doctor due to the risk of transmission of HIV, HCV, or HBV or other infectious diseases. Always follow the safety guidelines and procedures of your facility.
3. To prevent needlestick injuries, never put fingers in a needle holder.
4. The product is intended for single use. Dispose of all sharp objects (needles) for blood collection in suitable containers for hazardous biological waste.
5. Dispose of used and contaminated needles and blood collection accessories such as the needle holder in suitable disposal containers for biological hazardous substances.
6. Do not use the product after the expiry date. The expiry date of the product is the last day of the month and year indicated.

Storage

Store the product at room temperature.

Limitations

Vacuum tubes with a cap diameter of ≤ 18.2 mm can be used.

Collecting and handling specimens

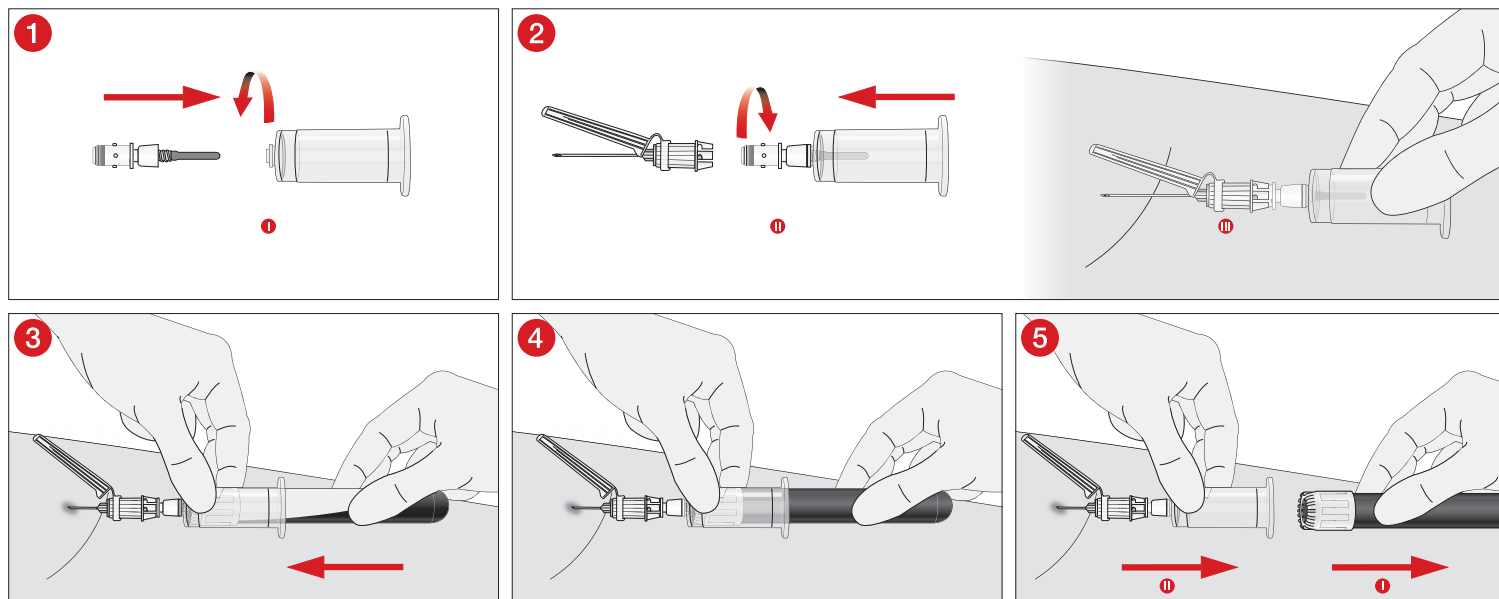
READ THIS DOCUMENT COMPLETELY.

Equipment required for specimen collection

1. All required vacuum tubes.
2. Blood culture adapter (Art. no. 14.1110), S-Monovette® (safety) needle or (Safety) Multify® needle and needle holder.
3. Gloves, gown, goggles or other suitable protective gear to protect against bloodborne pathogens or potentially infectious materials.
4. Labels for patient identification.
5. Disinfectant to clean the collection site (follow the guidelines of the facility for sterile specimen collection for preparing the collection site). Do not use any alcohol-based cleaning materials if the specimens are to be used for a blood alcohol test.
6. Dry, clean, disposable swabs.
7. A tourniquet.
8. Plaster or dressing.
9. Sharps disposal container for the safe disposal of used material.

Blood collection with the needle holder

BLOOD COLLECTION IS SHOWN USING THE S-MONOVETTE® SAFETY NEEDLE AS AN EXAMPLE.

1 + 2 For blood collection, the blood culture adapter is screwed into the needle holder **1** and then connected to the safety needle **2**. Puncture the vein with the safety needle with the needle holder attached **3**.

3 Then push the vacuum tube into the needle holder until the rubber membrane in the cap of the vacuum tube is pierced.

4 Wait until the blood stops flowing to allow correct filling.

5 The vacuum tube is disconnected from the safety needle with attached needle holder by pulling it out of the needle holder **5**. The safety needle with the attached vacuum needle holder remains in the vein. For multiple blood draws, repeat steps **1 - 5**.

Completing blood collection:

Pull the vacuum tube out of the needle holder to disconnect it **1**, then pull the safety needle with attached needle holder out of the vein **2** and use one hand to activate the needle guard according to the needle instructions for use. Invert each vacuum tube once immediately after disconnecting it from the safety needle, and after completing blood collection, carefully invert all specimens with anticoagulants several times.

1. Press a clean, dry swab against the puncture site until bleeding stops.
2. As soon as coagulation has set in, you may apply a dressing.
3. Dispose of the safety needle with attached needle holder in a suitable container for hazardous biological waste.

Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. Contaminated or filled blood collection systems must be disposed of in suitable containers for hazardous biological waste that can be subsequently autoclaved and incinerated.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).

Product-specific standards and guidelines as amended/updated

CLSI* GP41 "Collection of Diagnostic Venous Blood Specimens", Approved Standard.

*CLSI (Clinical and Laboratory Standards Institute)

Key for symbols and labels:

	Article number
	Batch number
	Use by
	CE mark
	<i>In vitro</i> diagnostic device
	Follow the instructions for use
	If reused: Risk of contamination
	Keep away from sunlight
	Store in a dry place
	Manufacturer
	Country of manufacture

Technical modifications reserved.

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

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