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SARSTEDT UriSet 24

DE	Gebrauchshinweis – SARSTEDT UriSet 24	2 - 6
EN	Instructions for Use – SARSTEDT UriSet 24	7 - 11
BG	Инструкции за употреба – SARSTEDT UriSet 24	12 - 16
CS	Návod k obsluze – SARSTEDT UriSet 24	17 - 21
DA	Brugsanvisning – SARSTEDT UriSet 24	22 - 26
EL	Οδηγίες χρήσεως – SARSTEDT UriSet 24	27 - 31
ES	Instrucciones de uso – SARSTEDT UriSet 24	32 - 36
ET	Kasutusjuhend – SARSTEDT UriSet 24	37 - 41
FR	Mode d'emploi – SARSTEDT UriSet 24	42 - 46
HR	Uputa za upotrebu – SARSTEDT UriSet 24	47 - 51
HU	Használati utasítás – SARSTEDT UriSet 24	52 - 56
IT	Istruzioni d'uso – SARSTEDT UriSet 24	57 - 61
KO	사용 설명서 – SARSTEDT UriSet 24	62 - 66
LT	Naudojimo instrukcijos – SARSTEDT UriSet 24	67 - 71
LV	Lietošanas norādes – SARSTEDT UriSet 24	72 - 76
NL	Gebruiksaanwijzing – SARSTEDT UriSet 24	77 - 81
NO	Bruksanvisning – SARSTEDT UriSet 24	82 - 86
PL	Instrukcja obsługi – zestaw UriSet 24 SARSTEDT	87 - 91
PT	Instruções de utilização – UriSet 24 SARSTEDT	92 - 96
RO	Instrucțiuni de utilizare – UriSet 24 SARSTEDT	97 - 101
RU	Инструкция по применению – SARSTEDT UriSet 24	102 - 106
SK	Návod na Použitie – SARSTEDT UriSet 24	107 - 111
SL	Navodila za uporabo – UriSet 24 SARSTEDT	112 - 116
SV	Bruksanvisning – SARSTEDT UriSet 24	117 - 121
TH	คำแนะนำในการใช้งาน – ชุดเก็บตัวอย่างบัสสาวะ UriSet 24 ของ SARSTEDT	122 - 126
TR	Kullanım için talimatlar – SARSTEDT UriSet 24	127 - 131
ZH	使用说明 – SARSTEDT UriSet 24	132 - 136

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DIN EN 14254: *In-vitro* diagnostic medical devices - Single-use receptacles for the collection of specimens, other than blood, from humans
 CLSI* GP16 "Urinalysis Approved Guideline"
 *CLSI (Clinical and Laboratory Standards Institute)

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Intended use


The UriSet 24 is used as a specimen collection vessel for the collection, transport, storage and processing (e.g. centrifugation) of urine for *in vitro* diagnostic tests. The product is intended for use by patients (specimen collection) under the instruction of qualified medical personnel and in a professional environment by qualified medical and laboratory personnel.

Product description

The UriSet 24 is available in several variants:

UriSet 24	Components	Quantity
A UriSet 24 with stabiliser and transport tube 30 ml (with light protection)	a Urine container 3 l b 500 ml collection cup c Transport tube d Stabiliser e Self-adhesive patient label Instructions for Use	1 1 1 1 1
B UriSet 24 with stabiliser and Urine Monovette® Z (with light protection)	a Urine container 3 l b 500 ml collection cup d Stabiliser e Self-adhesive patient label f Urine Monovette® Z g Suction tip Instructions for Use	1 1 1 1 1 1
C UriSet 24 without stabiliser and Urine Monovette® Z (with light protection)	a Urine container 3 l b 500 ml collection cup e Self-adhesive patient label f Urine Monovette® Z g Suction tip Instructions for Use	1 1 1 1 1

Product overview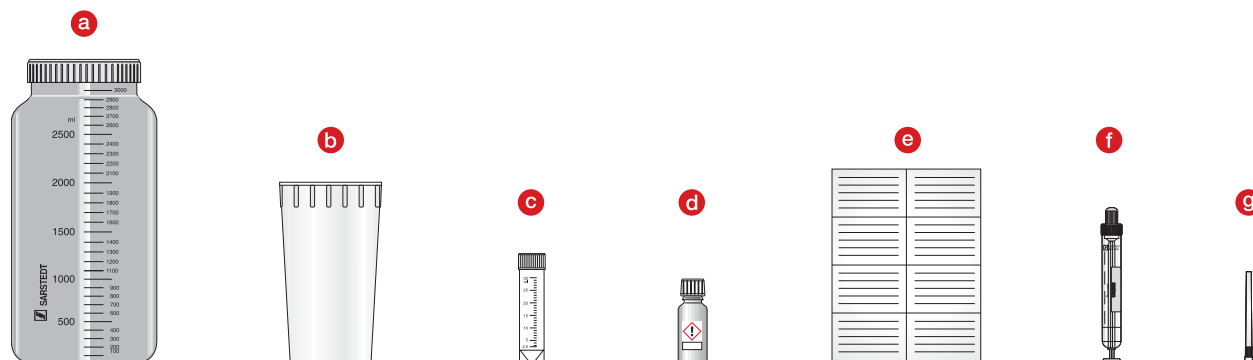

a Urine container

The urine container of the UriSet 24 is suitable for a filling volume up to three litres and consists of a brown plastic vessel (with light protection) with a wide opening diameter, recessed grips and a matching screw cap. The filling volume can be read through a clear viewing strip with a graduated scale.

b 500 ml collection cup

The collection cup for a volume of 500 ml is made of plastic.

c Transport tube 30 ml

The transport tube for a volume of 30 ml consists of white plastic with a printed graduated scale and matching screw cap. It does not contain any stabilising additives. The product is non-sterile.

d Stabiliser

The stabiliser comes in a brown glass bottle with a screw cap and safety dropper and contains nine millilitres of 20 % hydrochloric acid (HCl). The quantity and concentration are selected so that the catecholamines adrenalin, noradrenalin and dopamine and their metabolites metanephrine, normetanephrine, vanillylmandelic acid, homovanillic acid and 5-hydroxyindoleacetic acid are stabilised at room temperature for up to 7 days. Information on product-specific literature and studies is available on the SARSTEDT homepage: www.sarstedt.com/en/products/diagnostic/urine/bottles/

e Self-adhesive patient label

A white sheet of labels with black print has eight individual labels measuring 62x40 mm for handwritten labelling of the patient specimen.

f Urine Monovette® Z

The Urine Monovette® Z with light protection consists of a white plastic container with a transparent label, a conical plunger with plunger rod, a yellow Luer screw cap with stopper and an enclosed suction tip. It is suitable for a fill volume of up to 10 ml. It does not contain any stabilising additives. The product is non-sterile.

g Suction tip for Urine Monovette® Z

The plastic suction tip is an accessory for collecting urine in a Urine Monovette® Z.

Safety information and warnings

1. General precautions: Use gloves and other general personal protective equipment to protect yourself from urine and possible exposure to pathogens transmitted by biological samples.
2. Handle all biological specimens and sharp collection utensils (needles) according to the guidelines and procedures of your facility.
In case of direct contact with biological specimens or of a needlestick injury, consult a doctor due to the risk of transmission of HIV, HCV, or HBV or other infectious diseases.
Always follow the safety guidelines and procedures of your facility.
3. Only for variants containing a stabiliser. **Warning: The stabiliser contains hydrochloric acid.**
Irritates eyes, respiratory organs and skin. Keep closed and out of the reach of children.
In case of contact with the eyes, rinse thoroughly with water and consult a doctor.
In case of an accident or discomfort, consult a doctor immediately (show the product label if possible).
4. Always store the filled urine container in an upright position, closed and out of the reach of children.
5. The products are intended for single use. Dispose of the products and all collection utensils in containers for hazardous biological waste.
6. Do not use the product after the expiry date. The expiry date of the product is the last day of the month and year indicated.

Storage

Store the products at room temperature.

Transport

The transport tube and Urine Monovette® Z are a primary receptacle according to the ADR (Packing instruction P650) and IATA regulations.

Limitations

1. The appropriate storage and stabilisation of the urine during the 24-hour collection period depend on the parameters to be examined. Note the information from the laboratory.
2. The suitability of the urine containers for the analysis of trace elements/metals has not been tested.

Collecting and handling specimens

READ ALL OF THIS DOCUMENT BEFORE YOU START URINE COLLECTION.

Equipment required for specimen collection


- UriSet 24

Procedure for a 24-hour urine collection

A: When using a UriSet 24 with stabiliser and transport tube 30 ml

Empty your bladder in the toilet in the morning when you get up. Note the date and time on the enclosed label **e**.

1. Use the collection cup **b** to collect the next urine and transfer it to the container **a**.
2. Then drip the entire contents (acid) of the glass vial **d** into the container **a**.
Warning: The glass vial contains 9 ml hydrochloric acid 20% – follow safety instructions!
3. Close the container **a** firmly and swirl it back and forth. Collect all further urine, even when having a bowel movement, and put it in the container **a**.
Close the container **a** firmly every time and swirl it back and forth. Go to the toilet the next day at the time noted on the day before, collect this urine and add it to the container. Collection is then ended.
4. Read the collected quantity on the graduated scale on the container **a**. Write this number on the label **e**. Fill out the label **e** completely.
5. Put some urine from the container **a** into the collection cup **b**.
6. Fill the transport tube **c** with urine up to the 30 ml mark. Then empty the rest of the urine from the cup **b** and the container **a** into the toilet.
7. Close the transport tube **c**. Apply the completed label **e**. Bring the filled transport tube **c** to your doctor.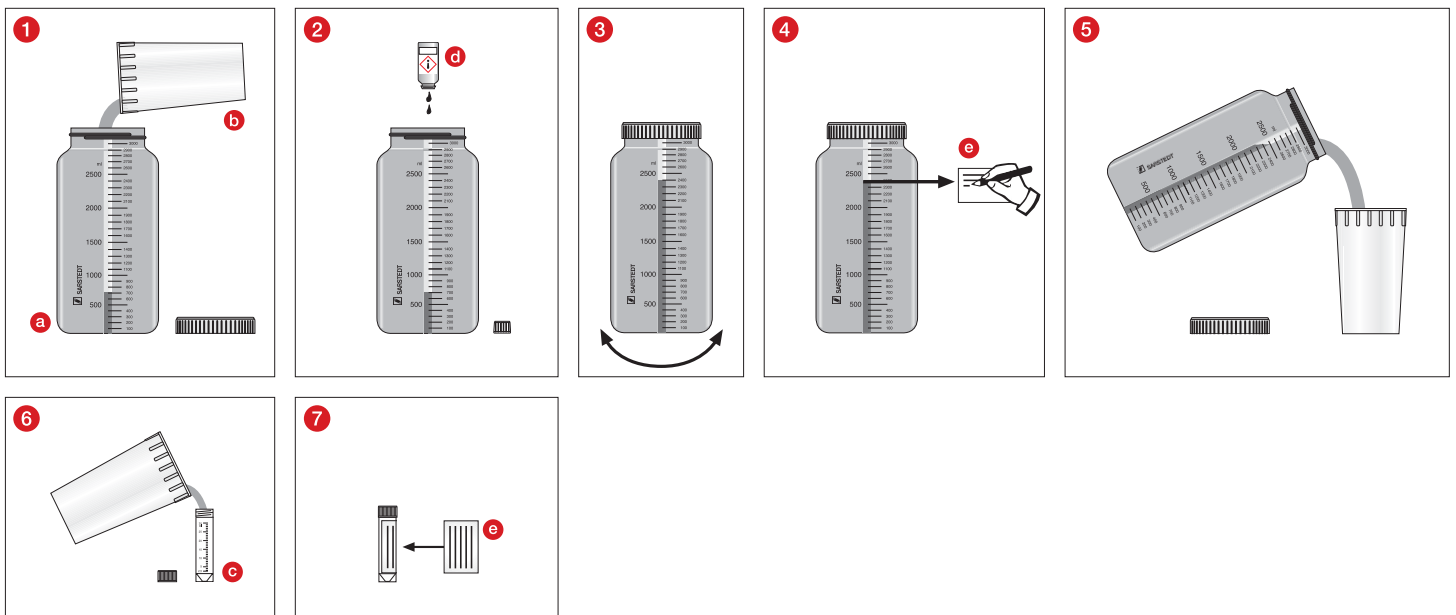


a: Container, **b:** Collection cup, **c:** Transport tube 30 ml, **d:** Glass vial with 9 ml hydrochloric acid 20%, **e:** Label for transport tube 30 ml:

B: When using a UriSet 24 with stabiliser and white Urine Monovette® Z

Empty your bladder in the toilet in the morning when you get up. Note the date and time on the enclosed label **e**.

1. Use the collection cup **b** to collect the next urine and transfer it to the container **a**.
2. Then drip the entire contents (acid) of the glass vial **d** into the container **a**.
Warning: The glass vial contains 9 ml hydrochloric acid 20% – follow safety instructions!
3. Close the container **a** firmly and swirl it back and forth. Collect all further urine, even when having a bowel movement, and put it in the container **a**.
Close the container **a** firmly every time and swirl it back and forth. Go to the toilet the next day at the time noted on the day before, collect this urine and add it to the container. Collection is then ended.
4. Read the collected quantity on the graduated scale on the container **a**. Write this number on the label **e**. Fill out the label **e** completely.
5. Put some urine from the container **a** into the collection cup **b**.
6. Remove the stopper of the Urine Monovette® Z **f** and keep it in a safe place. Attach the suction tip **g** to the Urine Monovette® Z **f**.
7. Dip the suction tip **g** into the collection cup **b** and fill the Urine Monovette® Z **f** up to the base line.
8. To empty the suction tip **g**, hold the Urine Monovette® Z **f** upright outside the collection cup **b** with the tip pointing upwards. Pull the plunger rod downwards to the stop.
9. Remove the suction tip **g** and put the stopper back on, then break off the plunger rod.
Empty the rest of the urine from the cup **b** and the container **a** into the toilet.
10. Apply the completed label **e** to the Urine Monovette® Z **f**. Bring it **f** to your doctor.
11. If you were given multiple Urine Monovettes **f**, fill and label all of them and bring them to your doctor.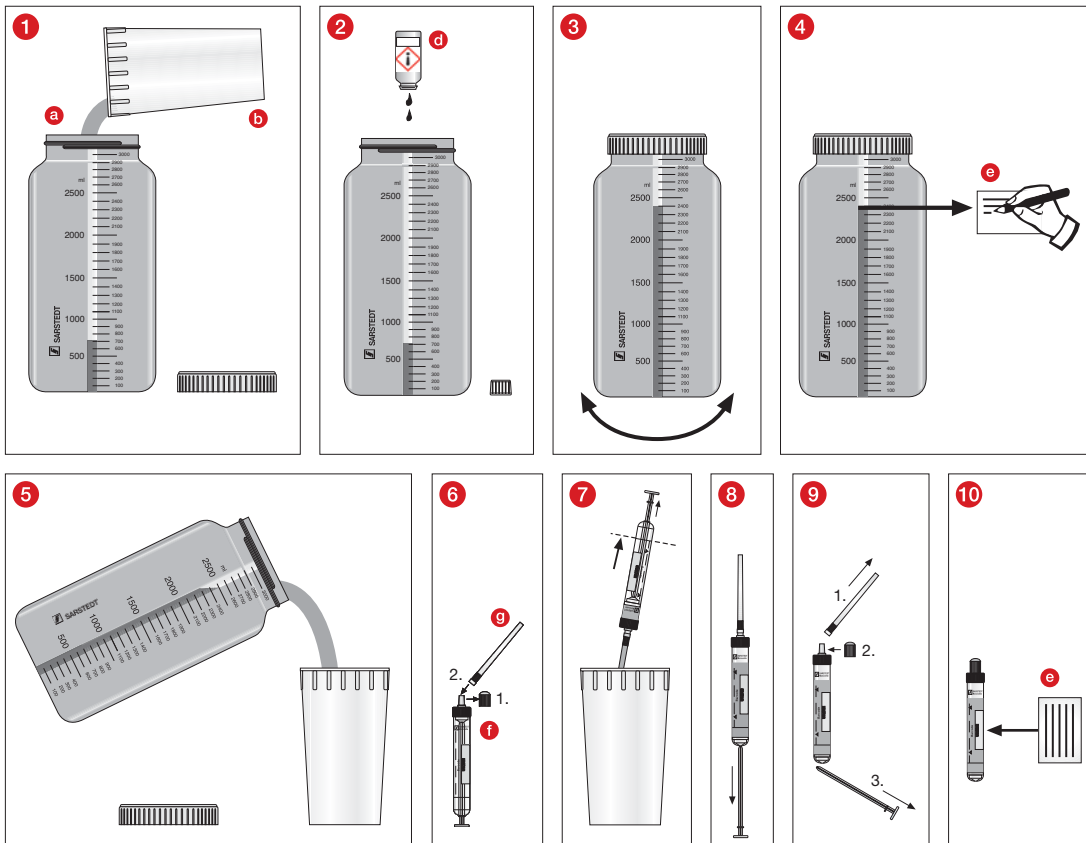


a: Container, **b**: Collection cup, **d**: Glass vial with 9 ml hydrochloric acid 20%, **e**: Label for Urine Monovette® Z, **f**: Urine Monovette® Z, **g**: Suction tip

C: When using a UriSet 24 with Urine Monovette® Z without stabiliser

Empty your bladder in the toilet in the morning when you get up. Note the date and time on the enclosed label **e**.

1. Use the collection cup **b** to collect the next urine and transfer it to the container **a**. Collect all further urine, even when having a bowel movement, and put it in the container **a**. Go to the toilet the next day at the time noted on the day before, collect this urine and add it to the container. Collection is then ended.
2. Read the collected quantity on the graduated scale on the container **a**. Write this number on the label **e**. Fill out the label **e** completely.
3. Swirl the closed container **a** several times until the urine is well mixed.
4. Put some urine from the container **a** into the collection cup **b**.
5. Remove the stopper of the Urine Monovette® Z **f** and keep it in a safe place. Attach the suction tip **g** to the Urine Monovette® Z **f**.
6. Dip the suction tip **g** into the collection cup **b** and fill the Urine Monovette® Z **f** up to the base line.
7. To empty the suction tip **g**, hold the Urine Monovette® Z **f** upright outside the collection cup **b** with the tip pointing upwards. Pull the plunger rod downwards to the stop.
8. Remove the suction tip **g** and put the stopper back on, then break off the plunger rod. Empty the rest of the urine from the cup **b** and the container **a** into the toilet.
9. Apply the completed label **e** to the Urine Monovette® Z **f**. Bring the filled Urine Monovette® Z **f** to your doctor.
10. If you were given multiple Urine Monovettes **f**, fill and label all of them and bring them to your doctor.

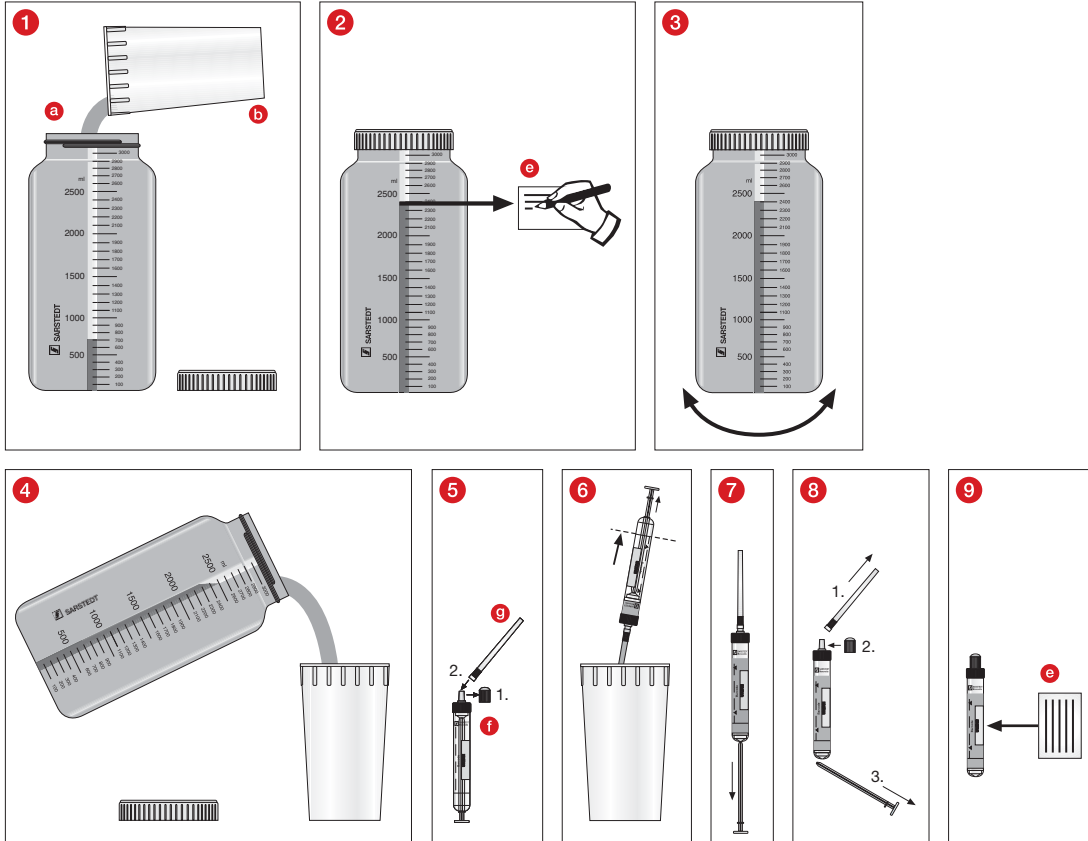a: Container, **b**: Collection cup, **e**: Label for Urine Monovette® Z, **f**: Urine Monovette® Z, **g**: Suction tip

Centrifugation

WARNING! SARSTEDT Urine Monovettes are designed for maximum 3000 x g (RCF – relative centrifugal force – g-force).

Centrifuge inserts should be selected according to the size of the Urine Monovettes used. The relative centrifugal force has the following ratio to the selected rpm:

$$\text{RCF (g-force)} = 11.2 \times r \times (\text{RPM}/1000)^2$$

“RCF”: “Relative centrifugal force”,

“RPM”: “Revolutions per minute” (min⁻¹),

“r” [in cm]: “Radius of rotation” from the middle of the centrifuge to the bottom of the Urine Monovette® Z.

Only use suitable tube holders or inserts. Centrifuging Urine Monovettes with cracks or centrifuging at an excessive centrifugal force can lead to breakage of the Urine Monovettes, which can release potentially hazardous substances.

Urine Monovettes should be centrifuged in accordance with the centrifugation conditions listed below. If other conditions are used, they must be validated by the user themselves.

It must be ensured that the Urine Monovettes fit well in the centrifuge inserts. Urine Monovettes that protrude above the insert can get caught in the centrifuge head and be broken. Ensure that the centrifuge is filled evenly. Please refer to the centrifuge instructions for use for further information.

CAUTION! Do not remove broken Urine Monovettes by hand.

Refer to the centrifuge instructions for use for information on disinfecting the centrifuge.

Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. The products must be disposed of in suitable containers for hazardous biological waste that can be subsequently autoclaved and incinerated.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).
5. Please take note of the information in the safety data sheet.

Product-specific standards and guidelines as amended / updated

DIN EN 14254: *In vitro* diagnostic medical devices – Single-use receptacles for the collection of specimens, other than blood, from humans
 CLSI* GP16 "Urinalysis Approved Guideline"
 *CLSI (Clinical and Laboratory Standards Institute)

Further literature:

European Urinalysis Guidelines: Scand J. Clin. Lab. Invest 2000; 60: 1–96.

Key for symbols and labels:

	Article number
	Batch number
	Use by
	CE marking
	<i>In vitro</i> diagnostic device
	Follow the instructions for use
	If reused: Risk of contamination
	Store away from sunlight
	Store in a dry place
	Manufacturer
	Country of manufacture
	Warning

Technical modifications reserved.

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

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