
Instructions for Use

Medisponge

SARSTEDT No.: 54.9912.100



1. Remove sponge and fingernail cleaner from wrapper.
2. Clean fingernails with fingernail cleaner provided.
3. Fill sponge under slowly running stream of water.
4. Commence scrubbing action on hands for 20-30 seconds, wetting sponge intermittently under running water.
5. Wet arms to elbows, apply sponge and follow normal scrubbing procedure, taking care to maintain a good lather, only applying enough water to maintain this lather.
6. Wash and repeat so that 3 minutes of scrubbing time elapses.
7. Discard sponge to waste receptacle.
8. Rinse hands and forearms under running water, taking care to hold hands above the height of the elbows.
9. For Theatre Use – Dry with sterile towel, using fresh “sterile” area of towel for each hand and arm.
10. For Wards – Use paper towel and discard.