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Nabelklemme

DE	Gebrauchshinweis – Nabelklemme	2
EN	Instructions for use – Navel clamp	2
BG	Инструкции за употреба – Пълна клапа	3
CS	Návod k obsluze – Pupočnicková svorka	3
DA	Bruksanvisning – Navleklemme	4
EL	Οδηγίες χρήσεως – Ομφαλόδεμα	4
ES	Instrucciones de uso – Pinza de cordón umbilical	5
ET	Kasutusjuhend – Nabaklamber	5
FI	Käyttöohje – Navan puristin	6
FR	Mode d'emploi – Clamp ombilical	6
HR	Uputa za upotrebu – Stezaljka za pupkovinu	7
HU	Használati utasítás – Köldökcsat	7
IT	Istruzioni d'uso – Morsetto ombelicale	8
KO	사용 설명서 – 배꼽 폐색기	8
LT	Naudojimo instrukcijos – Virkštelės spaustukas	9
LV	Lietošanas norādes – Nabassaites skava	9
NL	Gebruiksaanwijzing – Navelklem	10
NO	Bruksanvisning – Navleklemme	10
PL	Instrukcja obsługi – Zaciskacz do pępowiny	11
PT	Instruções de utilização – clamp umbilical	11
RO	Instrucțiuni de utilizare – Clemă ombilicală	12
RU	Инструкция по применению – Зажим для пуповины	12
SK	Návod na použitie – Pupočnicková svorka	13
SL	Navodila za uporabo – Sponka za popkovino	13
SV	Bruksanvisning – Navelklämna	14
TH	คำแนะนำในการใช้งาน – อุปกรณ์หนีบสายสะดือ	14
TR	Kullanım için talimatlar – Göbek klampı	15
ZH	使用说明 – 脐带夹 货号	15

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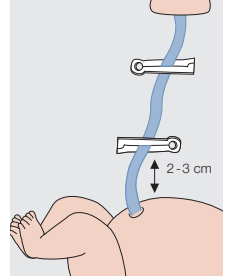
EN: Instructions for Use - Navel clamp

Designation of purpose

The Navel clamp is used during the birth of children. It is used to clamp the umbilical cord before the cord is cut.

Product specification

The navel clamp is made of high-quality plastic and has a safety lock that cannot be opened after it has been engaged once. The navel clamp is in a blister, single sterile packed and only suitable for single use.

Product handling

The cord should be clamped about 2 cm above the navel ring with the navel clamp. The umbilical cord should also be clamped in the direction of the placenta. It is important that the navel clamps are firmly closed and audibly click into place. If the clamp is closed properly, it cannot be opened without tools. Before the baby is wrapped in cloths or dressed, the position of the clamp must be checked again. If the navel has already dried up so much, the navel clamp can be removed by dropping off the rest of the navel cord or opening the navel clamp with a suitable tool.

Storage

The navel clamps should be stored at room temperature and avoid exposure to direct sunlight. Do not use the navel clamps after the expiry date has passed.

Disposal

After use, the navel clamp can be disposed of in normal household waste.

Safety instructions/warnings

- do not use if packaging is opened or damaged
- always ensure correct closure/engagement of the clamp
- Do not remove until the navel is completely dry
- Use only by qualified personnel

 Do not use if package is damaged

 Risk of contamination if reused

 SARSTEDT AG & Co. KG
Sarstedtstr. 1
D-51588 Nümbrecht
www.sarstedt.com

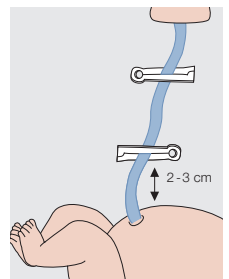
All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

Store at room temperature

Technical modifications reserved

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