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Produktspezifische Normen:

DIN EN 14254: In vitro diagnostic medical devices - Single-use receptacles for the collection of specimens, other than blood, from humans
 CLSI* GP16 "Urinalysis Approved Guideline"
 *CLSI (Clinical and Laboratory Standards Institute)

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DIN EN 1041: Information supplied by the manufacturer of medical devices
 DIN EN ISO 15223-1: Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements

Normen zu Sterilisation:

DIN EN ISO 11137: Sterilization of health care products - Radiation

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DIN EN ISO 11607: Packaging for terminally sterilized medical devices

Systembezogene Normen:

DIN EN ISO 13485: Medical devices - Quality management systems – Requirements for regulatory purposes
 MDSAP: Medical Device Single Audit Program gemäß ISO 13485
 DIN EN ISO 14971: Medical devices - Application of risk management to medical devices
 IVDR (EU) 2017/746: Verordnung über In-vitro-Diagnostika

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 SARSTEDT AG & Co. KG
 Sarstedtstr. 1
 D-51588 Nümbrecht
 www.sarstedt.com

Instructions for use – SARSTEDT V-Monovette® Urine with boric acid

For in vitro diagnostic use

EN

Intended use

The V-Monovette® Urine with boric acid is a disposable urine tube intended for collecting, storing and transporting urine specimens for microbiological tests in in vitro diagnostics. Bacteria and yeast cultures can be stored at room temperature for up to 48 hours before starting the culture. The product is intended for use in a professional environment by medically trained personnel.

Product description

The V-Monovette® Urine with boric acid tube consists of polyethylene terephthalate (PET) and is delivered with a defined vacuum corresponding to the nominal volume. The V-Monovette® Urine with boric acid has a green V-Monovette® closing cap and contains a preservative. This stabilises bacteria and yeast cultures at room temperature for up to 48 hours, corresponding to storage of the urine specimen at 4°C. The average concentration of the preservative (boric acid) in the urine specimen of a V-Monovette® Urine with boric acid is 1.5%.

Safety instructions and warnings

1. General precautions: Wear gloves and general personal protective equipment to protect yourself from possible exposure to potentially infectious specimen material and transmitted pathogens.
2. For single use only.
3. Handle all biological specimens and collection utensils according to the guidelines and procedures in your institution. In case of exposure to biological specimens or a needle prick injury, consult a doctor, as infectious diseases can be transmitted.
4. Dispose of all collection utensils in containers for hazardous biological waste.
5. Do not use the product after the expiry date. The expiry date of the V-Monovette® Urine is the last day of the month and year indicated.
6. The product may be used only by medically trained personnel following these instructions for use.
7. To prevent needle prick injuries, never put fingers in a urine transfer unit or urine container with integrated transfer unit.
8. Removing the cap of the V-Monovette® to fill the V-Monovette® Urine with boric acid impairs its sterility and is not recommended.
9. The V-Monovette® Urine with boric acid already contains boric acid. Please observe the safety data sheet.
<https://www.sarstedt.com/en/download/safety-data-sheets/>

Procedure for transferring the urine specimen to a V-Monovette® Urine with boric acid

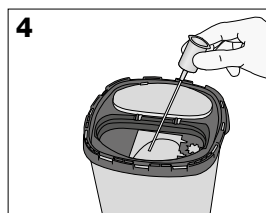
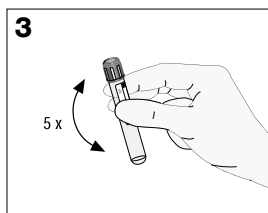
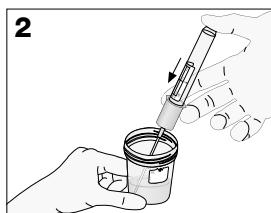
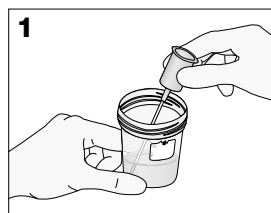
READ THIS DOCUMENT COMPLETELY BEFORE STARTING A URINE TRANSFER.

Material needed for the urine transfer:

1. Gloves, gown, goggles or other suitable protective gear to protect against potentially infectious materials.
2. Required number of V-Monovettes Urine with boric acid.
3. Urine transfer unit if urine cups without integrated transfer unit are used.
4. Sharps disposal container for disposal of used urine transfer units.
5. Disposal container for hazardous biological waste.


A: When using the transfer unit:

1. Prepare the urine sample by carefully swivelling the urine collection cup.
If several specimens are to be filled, the specimens for microbiological tests should be taken first.
2. Open the urine cup. Dip the tip of the urine transfer unit into the urine specimen.
3. Guide the V-Monovette® Urine with the cap pointing downwards into the transfer unit and make sure that the needle penetrates the stopper of the V-Monovette® Urine. The V-Monovette® Urine starts filling automatically according to the defined vacuum. If no urine flows or the flow stops before a sufficient filling is achieved, the following steps are recommended:
 - a. Press the V-Monovette® Urine forward firmly again to make sure that the stopper is completely penetrated.
 - b. Secure the V-Monovette® Urine in the transfer unit during the filling process by pressing your thumb against the bottom of the V-Monovette® Urine.
 - c. If the urine still does not flow, remove the V-Monovette® Urine and use a new V-Monovette® Urine.
4. Hold the V-Monovette® Urine in place until no more urine flows into it.
5. Remove the V-Monovette® Urine from the transfer unit and only then take the transfer unit out of the fluid.
6. Swivel the V-Monovette® Urine with boric acid at least five times or until the specimen is completely mixed with the preservative.
7. Dispose of the transfer unit in a sharps disposal container.
8. Close the urine cup and dispose of it in a container for hazardous biological waste.

B: When using a urine cup with an integrated transfer unit:

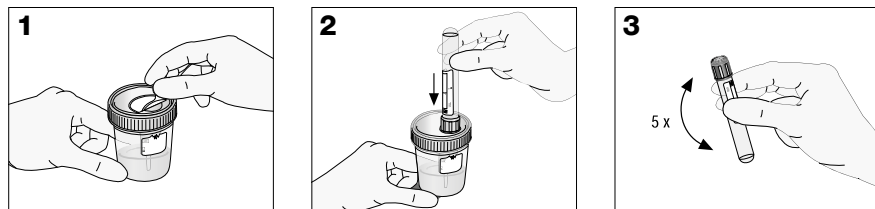1. Prepare the urine sample by carefully swivelling the urine collection cup. If several specimens are to be filled, the specimens for microbiological tests should be taken first.
2. Remove the safety label from the top of the urine cup to allow access to the integrated transfer unit. Stick the safety label back on the opening to seal it after collecting the urine.
NOTE: The minimum filling quantity required for filling a V-Monovette® Urine is 20 ml for the closed urine cup.
3. Guide the V-Monovette® Urine with the cap pointing downwards into the integrated transfer unit and make sure that the needle penetrates the stopper of the V-Monovette® Urine. The V-Monovette® Urine starts filling automatically according to the defined vacuum. If no urine flows or the flow stops before a sufficient filling is achieved, the following steps are recommended:
 - a. Press the V-Monovette® Urine forward firmly again to make sure that the stopper is completely penetrated.
 - b. Secure the V-Monovette® Urine in the transfer unit during the filling process by pressing against the bottom of the V-Monovette® Urine.
 - c. If the urine still does not flow, remove the V-Monovette® Urine and use a new V-Monovette® Urine.
4. Hold the V-Monovette® Urine with boric acid in place until no more urine flows into it and only then remove the tube from the integrated transfer unit.
5. Swivel the V-Monovette® Urine at least five times or until the specimen is completely mixed with the preservative.
6. Dispose of the urine cup in a container for hazardous biological waste.

**Features**

The V-Monovette® Urine with boric acid inhibits the growth of urinary pathogenic microorganisms in the urine at room temperature for up to 48 hours without impairing their viability. Studies were conducted for selected strains of bacteria:

Escherichia coli
Klebsiella pneumoniae
Proteus mirabilis
Streptococcus faecalis
Candida albicans

Information on the studies is available on the SARSTEDT website
[https://www.sarstedt.com/en/products/diagnostic/urine/urine-monovetter/ Literature / "Urine Monovette® with Boric Acid"](https://www.sarstedt.com/en/products/diagnostic/urine/urine-monovetter/ Literature /)

Limitations

The sample volume that is transferred to a V-Monovette® Urine with boric acid can vary depending on different physical factors such as altitude, ambient temperature, the remaining shelf life of the product and the filling procedure.
The V-Monovette® Urine with boric acid must be filled up to the filling line to ensure the necessary ratio of urine to preservative.
A deviation of +/-10% from the filling line is permissible. Underfilling can lead to faulty results.

Storage

The V-Monovette® Urine with boric acid can be stored at room temperature until it is used.

Recommended storage temperature after use

The filled V-Monovette® Urine with boric acid can be stored at room temperature for up to 48 hours.

Disposal

1. The general hygiene guidelines and statutory regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. Contaminated or filled V-Monovettes Urine must be disposed of in suitable containers for hazardous biological waste that can be subsequently autoclaved and incinerated.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).

US standards / ISO standards as last amended

Product-specific standards

DIN EN 14254: In vitro diagnostic medical devices - Single-use receptacles for the collection of specimens, other than blood, from humans
 CLSI* GP16 "Urinalysis Approved Guideline"
 *CLSI (Clinical and Laboratory Standards Institute)

Standards for labelling:

DIN EN 1041: Information supplied by the manufacturer of medical devices
 DIN EN ISO 15223-1: Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements

Standards for sterilisation:

DIN EN ISO 11137: Sterilization of health care products - Radiation

Standards for packaging:

DIN EN ISO 11607: Packaging for terminally sterilized medical devices

System-related standards:

DIN EN ISO 13485: Medical devices - Quality management systems – Requirements for regulatory purposes
 MDSAP: *Medical Device Single Audit Program* in accordance with ISO 13485
 DIN EN ISO 14971: Medical devices - Application of risk management to medical devices
 IVDR (EU) 2017/746: In Vitro Diagnostic Regulation

Additional literature:

European Urinalysis Guidelines: Scand J. Clin. Lab. Invest 2000; 60: 1–96.

Key for icons and labels:

- | | |
|--|--|
|  REF Article number |  Use before |
|  LOT Batch number |  Follow the instructions for use |
|  CE CE marking |  Store away from sunlight |
|  Manufacturer |  Store in a dry place |
| Store at room temperature |  Do not use if package is damaged |
|  STERILE R Sterilisation by irradiation |  IVD For in-vitro diagnostic use |
|  Risk of contamination if reused | |
|  Do not resterilise | |

Technical modifications reserved

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

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[https://www.sarstedt.com/en/products/diagnostic/urine/urine-monovetter/ Literature / "Urine Monovette" with Boric Acid](https://www.sarstedt.com/en/products/diagnostic/urine/urine-monovetter/ Literature /)

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Systemrelaterede standarder:

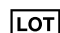
DIN EN ISO 13485: Medical devices - Quality management systems – Requirements for regulatory purposes
 MDSAP: *Single Audit Program for medicinsk udstyr* i henhold til ISO 13485
 DIN EN ISO 14971: Medical devices - Application of risk management to medical devices
 IVDR (EU) 2017/746: Forordning om in vitro-diagnosticering

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[https://www.sarstedt.com/en/products/diagnostic/urine/urine-monovetter/ Literature / "Urine Monovette® with Boric Acid"](https://www.sarstedt.com/en/products/diagnostic/urine/urine-monovetter/ Literature /)

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DIN EN 14254: In vitro diagnostic medical devices - Single-use receptacles for the collection of specimens, other than blood, from humans
 CLSI® GP16 "Urinalysis Approved Guideline"
 "CLSI (Clinical and Laboratory Standards Institute)

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DIN EN 14254: In vitro diagnostic medical devices – Single-use receptacles for the collection of specimens, other than blood, from humans
 CLSI* GP16 “Urinalysis Approved Guideline”
 *CLSI (Clinical and Laboratory Standards Institute)

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DIN EN 1041: Information supplied by the manufacturer of medical devices
 DIN EN ISO 15223-1: Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied – Part 1: General requirements

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DIN EN ISO 11137: Sterilization of health care products – Radiation

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DIN EN ISO 13485: Medical devices – Quality management systems – Requirements for regulatory purposes
 MDSAP: ISO 13485에 따른 Medical Device Single Audit Program
 DIN EN ISO 14971: Medical devices – Application of risk management to medical devices
 IVDR (EU) 2017/746: 체외 진단 도구에 관한 규정

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DIN EN 14254: In vitro diagnostic medical devices - Single-use receptacles for the collection of specimens, other than blood, from humans
 CLSI® GP16 "Urinalysis Approved Guideline"
 *CLSI (Clinical and Laboratory Standards Institute)

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DIN EN ISO 13485: Medical devices - Quality management systems – Requirements for regulatory purposes
 MDSAP: Medical Device Single Audit Program atbilstoši ISO 13485
 DIN EN ISO 14971: Medical devices - Application of risk management to medical devices
 IVDR (EU) 2017/746: Regula par in vitro diagnostiku

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[https://www.sarstedt.com/en/products/diagnostic/urine/urine-monovetter/Literature/\"Urine Monovette® with Boric Acid\"](https://www.sarstedt.com/en/products/diagnostic/urine/urine-monovetter/Literature/\)

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Standardi, ki se nanašajo na sistem:

DIN EN ISO 13485: Medical devices - Quality management systems – Requirements for regulatory purposes
 MDSAP: Medical Device Single Audit Program skladno z ISO 13485
 DIN EN ISO 14971: Medical devices - Application of risk management to medical devices
 IVDR (EU) 2017/746: Uredba o in vitro diagnostiki

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DIN EN 14254: In vitro diagnostic medical devices - Single-use receptacles for the collection of specimens, other than blood, from humans
CLSI* GP16 "Urinalysis Approved Guideline"
*CLSI (Clinical and Laboratory Standards Institute)

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DIN EN 1041: Information supplied by the manufacturer of medical devices
DIN EN ISO 15223-1: Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements

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DIN EN ISO 13485: Medical devices - Quality management systems – Requirements for regulatory purposes
MDSAP: ISO 13485 uyarınca *Medical Device Single Audit Program*
DIN EN ISO 14971: Medical devices - Application of risk management to medical devices
IVDR (EU) 2017/746: In vitro teşhis ile ilgili yönetmelik

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