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DE	Gebrauchshinweis – SARSTEDT NFT Urin-Entnahmesystem	1 - 11
EN	Instructions for Use – SARSTEDT NFT Urine Collection System	12 - 21
BG	Инструкции за употреба – SARSTEDT NFT система за вземане на урина	22 - 31
CS	Návod k obsluze – Systém pro odběr moči NFT SARSTEDT	32 - 41
DA	Brugsanvisning – SARSTEDT NFT urinprøvetagningssystem	42 - 51
EL	Οδηγίες χρήσεως – Σύστημα συλλογής ούρων SARSTEDT NFT	52 - 61
ES	Instrucciones de uso – Sistema de extracción de orina NFT SARSTEDT	62 - 71
ET	Kasutusjuhend – SARSTEDT'i uriini proovivõtusüsteem NFT	72 - 81
FR	Mode d'emploi – Système de prélèvement d'urine NFT de SARSTEDT	82 - 91
HR	Uputa za upotrebu – sustav za prikupljanje urina NFT	92 - 101
HU	Használati utasítás – SARSTEDT NFT vizelet mintavételi rendszer	102 - 111
IT	Istruzioni d'uso – Sistema di prelievo dell'urina NFT SARSTEDT	112 - 121
KO	사용 설명서 – SARSTEDT NFT 소변 수집 시스템	122 - 131
LT	Naudojimo instrukcijos – SARSTEDT NFT šlapimo surinkimo sistema	132 - 141
LV	Lietošanas norādes – SARSTEDT NFT urīna paņemšanas sistēma	142 - 151
NL	Gebruiksaanwijzing – SARSTEDT NFT urine-afnamesysteem	152 - 161
NO	Bruksanvisning – SARSTEDT NFT urin-prøvetakingssystem	162 - 171
PL	Instrukcja obsługi – System do pobierania moczu NFT firmy SARSTEDT	172 - 181
PT	Instruções de utilização – Sistema de colheita de urina NFT SARSTEDT	182 - 191
RO	Instrucțiuni de utilizare – Sistem de colectare a urinei NFT SARSTEDT	192 - 201
RU	Инструкция по применению – система сбора мочи NFT SARSTEDT	202 - 211
SK	Návod na Použitie – SARSTEDT NFT systém na odber moču	212 - 221
SL	Navodila za uporabo – Sistem za odvzem urina NFT SARSTEDT	222 - 231
SV	Bruksanvisning – SARSTEDT NFT system för urinprovtagning	232 - 241
TH	คำแนะนำในการใช้งาน – ระบบเก็บปัสสาวะ NFT ของ SARSTEDT	242 - 251
TR	Kullanım için talimatlar – SARSTEDT NFT İdrar Alma Sistemi	252 - 261
ZH	使用说明 – 萨斯特 (SARSTEDT) NFT 尿液采集系统	262 - 271

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Intended use

The SARSTEDT NFT Urine Collection System is used as a closed sample container system for needle-free transfer (NFT) of urine from an NFT urine cup or an NFT urine collection container to a Urine Monovette®. The NFT urine cup, the NFT urine collection container and the Urine Monovette® are used as sample containers for the collection, transport and storage of urine for *in vitro* diagnostic tests. The Urine Monovette® is also suitable for processing the urine sample (e.g. by centrifugation). The products are available individually and can be used together. The NFT Urine Collection System is intended for use in a professional environment by qualified medical and laboratory personnel.

Product description

The following individual components are available for the NFT Urine Collection System:

Urine Monovette®

Name	Colour code	Description
Containers without additive		
Urine Monovette® Z 3.2 ml	yellow	75x13 mm, round base
Urine Monovette® Z 8.5 ml		92x15 mm, round base
Urine Monovette® Z 10 ml		102x15 mm, round base
Containers with additive		
Urine Monovette® Boric acid 10 ml	green	102x15 mm, round base

Urine Monovette® Z

The Urine Monovette® Z, except for the version with light protection properties, consists of a transparent plastic container, a conical plunger with plunger rod, a coloured Luer screw cap with stopper and an enclosed suction tip. Depending on the version, the Urine Monovette® Z has either a plastic or paper label. In addition, various versions of the Urine Monovette® Z are available for different sample volumes. They do not contain any stabilising additives. The white Urine Monovette® Z with light protection properties, Art. no. 10.252.030, is especially suitable for protecting light-sensitive parameters. Sterile and non-sterile versions are available.

Urine Monovette® Boric acid


The Urine Monovette® Boric acid consists of a transparent plastic container, a conical plunger with plunger rod, a coloured Luer screw cap with stopper and an enclosed suction tip. The Urine Monovette® Boric acid is available optionally with a plastic or paper label and in a sterile or non-sterile version.

The product is prepared with boric acid. The average concentration of boric acid for a filled Urine Monovette® Boric acid is 1.5% and stabilises the growth of pathogenic microorganisms in the urine at room temperature for up to 48 hours without impairing their viability. Studies were conducted for selected strains of bacteria:

- *Escherichia coli*
- *Klebsiella pneumoniae*
- *Proteus mirabilis*
- *Streptokokkus faecalis*
- *Candida albicans*

Information on product-specific studies is available on the SARSTEDT homepage: www.sarstedt.com/en/products/diagnostic/urine/urine-monovetter/

NFT collection receptacles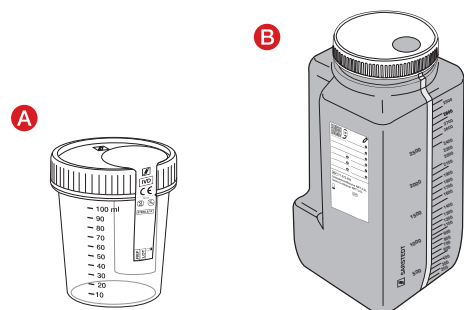

Type	Name	Description
A	NFT urine cup	Urine cup with an integrated NFT transfer unit
B	NFT urine collection container	Urine collection container with an integrated NFT transfer unit

NFT urine cup

The NFT urine cup with needle-free transfer unit is available for a fill volume of up to 100 ml and consists of a transparent plastic container, graduated and with a labelling field. A needle-free transfer unit with a penetrable NFT membrane is integrated into the screw cap, providing a tight connection for the closed transfer of the urine sample into a Urine Monovette®. An intact safety label affixed to the outside of the cup indicates that the urine cup has not been tampered with. The product is sterile.

NFT urine collection container

The NFT urine collection container with integrated transfer unit is available for a fill volume of 3000 ml and consists of a brown-coloured (light protection) graduated plastic container with a wide opening diameter, inspection strip and recessed grips. A needle-free transfer unit with a penetrable NFT membrane is integrated into the white screw cap, providing a tight connection for the closed transfer of the urine sample into a Urine Monovette®. The product is non-sterile.

Accessories for NFT urine collection container

Self-adhesive label for NFT urine collection container

Order no.	Dimensions	Version	Packaging
84.1402	150x55 mm	Label in German	300/inner carton (10x30/bag)
84.1402.001	150x55 mm	Label in English	300/inner carton (10x30/bag)
84.1402.002	150x55 mm	Label in French	300/inner carton (10x30/bag)
84.1402.003	150x55 mm	Label in Dutch	300/inner carton (10x30/bag)

Safety information and warnings

NOTE FOR STERILE VERSIONS: DO NOT USE THE PRODUCT IF THE BLISTER PACKAGING OR THE SAFETY LABEL IS DAMAGED.

1. General precautions: Use gloves and other general personal protective equipment to protect yourself from urine and possible exposure to pathogens transmitted by biological samples.
2. Handle all biological specimens according to the guidelines and procedures in your facility.
In case of direct contact with biological samples, consult a doctor due to the risk of transmission of HIV, HCV, or HBV or other infectious diseases.
3. Always follow the safety guidelines and procedures of your facility.
4. The products are intended for single use. Dispose of the products and all collection utensils in containers for hazardous biological waste.
5. Do not use the products after the expiry date. The expiry date of the product is the last day of the month and year indicated.
6. Only for the Urine Monovette® Boric acid: Please note, the product contains boric acid.

Hazard statements:

H360FD Can impair fertility. Can damage the foetus in the womb.

Safety statements:

P201 Obtain special instructions before use.

P308 + P313 In case of exposure or if affected: Seek medical advice/attention.

Please note the safety data sheets: www.sarstedt.com/en/download/safety-data-sheets/

Storage

Store the products at room temperature.

Transport

The Urine Monovette® is a primary receptacle according to the ADR (Packing instruction P650) and IATA regulations.

The NFT urine cup and NFT urine collection container can be transported together with an ADR-compliant (Packing instruction P650) secondary vessel, for example B 17 transport case, Art. no. 95.1717.

Limitations

1. Urine test strips belong to the most frequently used diagnostic accessories.
Before using a test strip, the manufacturer's information and possible limitations must be taken into account.
2. The first morning urine is best suited for urine diagnostics. In the ideal case, there should be at least 4 hours between the previous urination and collecting the urine specimen.
3. Midstream urine specimens are recommended, since they show lower contamination from urethral and/or surrounding flora compared to spontaneous urine.
Detection of bacteria (type and number) and possibly the result of a test strip analysis may be adversely influenced by e.g. genital and/or urethral contamination.
4. Where possible, the urine specimens should be collected prior to beginning antibacterial therapy.
5. The storage duration and temperature for urine specimens depend on the expiry date of the parameters for investigation.
Without additional stabilisation, urine analysis (with the exception of 24-hour urine collection) should be carried out within one to two hours.
6. The appropriate storage and stabilisation of the urine during a 24-hour collection period depend on the parameters to be examined.
The laboratory should provide precise information on this.
7. Cooling of spontaneous and midstream urine samples at 2–8°C is permitted and may prolong sample stability for some analytes.
There are exceptions for light-sensitive parameters, for example bilirubin and urobilinogen. Store away from light!
8. Cooling the urine sample at 2–8°C can impair urine microscopy due to precipitation of amorphous urates or phosphates.
9. Do not freeze urine specimens if they are to be used for urine sediment.
10. For microbiological culture diagnostics, if there is no possibility of transport or immediate sample processing, storing the urine either cooled at 2–8°C or in a Urine Monovette® Boric acid is recommended.
11. The suitability of the urine specimen for the analysis of trace elements/metals has not been tested.
12. Applicable only for the Urine Monovette® Boric acid:
NOTE: Filling up to the fill level mark ensures an optimal urine-preservative ratio. A deviation of +/-10% is allowed. It should be noted that at the start of urine collection, a few grains of the additive may leak out, however, this does not affect the recommended minimum amount of boric acid or the stabilising effect. The use of the stabilised urine sample for strip tests and other non-microbiological analysis methods has not been tested and may lead to false results.

Collecting and handling specimens

READ ALL OF THIS DOCUMENT BEFORE YOU START URINE COLLECTION AND/OR URINE TRANSFER.

Equipment required for urine collection

A: NFT urine cup


1. NFT urine cup
2. Clean cloths or flannels and towel

B: NFT urine collection container

1. NFT urine collection container with enclosed patient label
2. If needed, collection cup for urine portions, for example 500 ml cup, Art. no. 75.1356.

A: Procedure for collecting midstream urine using an NFT urine cup


- 1 Wash your hands thoroughly and then the genital area. Dry yourself off with a clean towel.
- 2 Ensure that the safety label is intact.
- 3 Then open the lid of the urine cup by turning it anticlockwise. Place the lid in a hygienic place. Ensure that the inside of the lid is not contaminated.
- 4 First pass a small quantity of urine into the toilet (Fig. 4a), then catch the midstream urine in the urine cup without interrupting the urine flow (Fig. 4b). Then fully empty your bladder into the toilet (Fig. 4c).
- 5 Close the cap of the urine cup tightly by turning it clockwise. Be careful not to contaminate the inside of the lid. Label the urine cup with your name and immediately deliver the tightly closed urine cup with the specimen to the responsible person, or proceed as instructed by your doctor.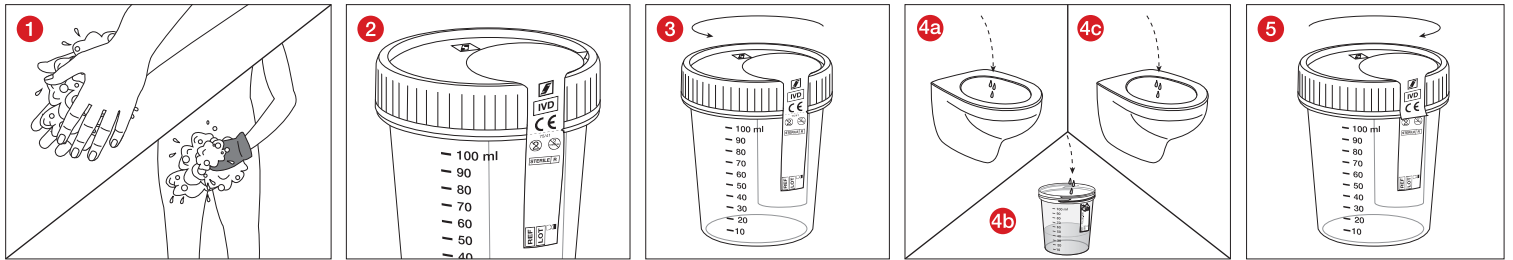

B: Procedure for a 24-hour urine collection using an NFT urine collection container

Multilingual directions for patients for collecting 24-hour urine can be obtained if necessary via a QR code on the product label and at www.sarstedt.com/urine-container-nft.

- 1 Empty your bladder into the toilet in the morning when you get up. (Fig. 1a). Note the date and time under 6 and 7 on the container label (Fig. 1b).
- 2 Collect your next urine in the NFT urine collection container. You may use a small collection cup, but one that is sufficiently large, and transfer all the urine to the collection container. Collect all further urine, including when defecating, and put it in the NFT urine collection container. Close the NFT urine collection container well after any additions.
- 3 Go to the toilet the next day at the time noted on the day before and collect this urine. Collection is then complete.
- 4 Read the collected quantity on the graduated scale on the NFT urine collection container. Enter this number under 1 on the container label and add the date and time of the most recent addition under 8 and 9.
- 5 Enter complete patient data under 2 – 5. Bring the filled NFT urine collection container to your doctor or proceed as instructed by your doctor.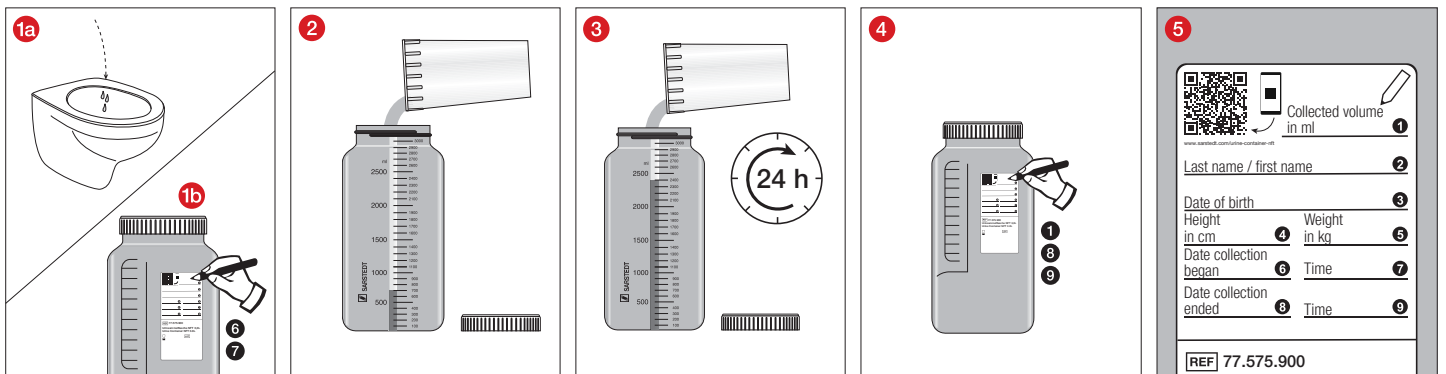

Procedure for transferring the urine sample to a Urine Monovette®

Material required for the urine transfer

1. Gloves, gown, goggles or other suitable protective wear to protect against potentially infectious materials.
2. Required number of Urine Monovettes®
3. Only required if the quantity of collected urine in an NFT urine collection container is less than 400 ml. Urine cup (e.g. Art. no. 75.562.005).
4. Stand or other suitable device for placing the filled Urine Monovettes.
5. Disposal container for hazardous biological waste.


Recommended order of collection

If multiple Urine Monovettes are to be filled, the following order of collection is recommended:

Urine Monovette®	Intended use
1. Urine Monovette® Z (without stabiliser)	Microbiological tests
2. Urine Monovette® Boric acid (with stabiliser)	
3. Urine Monovette® Z (without stabiliser)	Chemical tests, strip tests and urine particle analysis


A1: From an NFT urine cup with a fill volume ≥ 20 ml

1. Prepare the urine sample for removal by gently swirling the NFT urine cup (Fig. 1a).
NOTE: Make sure that the fill volume of the closed urine cup is 20 ml or more. If not, proceed as described under A2. Remove the safety label from the NFT urine cup to allow access to the needle-free transfer unit (Fig. 1b).
2. Only for individually sterile-wrapped Urine Monovettes: Open the blister package with the thumb and index finger using the peel-off technique. The transparent film must be facing upwards (Fig. 2a). Hold the Urine Monovette® upright with the tip pointing upwards and remove the stopper on the Luer screw cap. Set it aside for later (Fig. 2b).
3. First guide the Urine Monovette® with the Luer connector into the needle-free transfer unit and make sure that the Luer cone fully penetrates the membrane and the Urine Monovette® is firmly in place.
4. Use the plunger rod to pull the plunger to the bottom of the Urine Monovette® and wait until no more urine flows.
5. Detach the Urine Monovette® from the needle-free transfer unit and turn it upright so that the tip points upwards.
6. Close the Urine Monovette® securely with the previously removed stopper (Fig. 6a).
Only when using a Urine Monovette® 3.2 ml: Withdraw the plunger rod until it audibly clicks into the final position (Fig. 6b).
7. Break off the plunger rod.
8. Only when using a Urine Monovette® Boric acid: Invert the Urine Monovette® Boric acid around 5 times or until the boric acid is completely dissolved.
9. Place the Urine Monovette® in a stand.
10. Dispose of the NFT urine cup in a container for hazardous biological waste.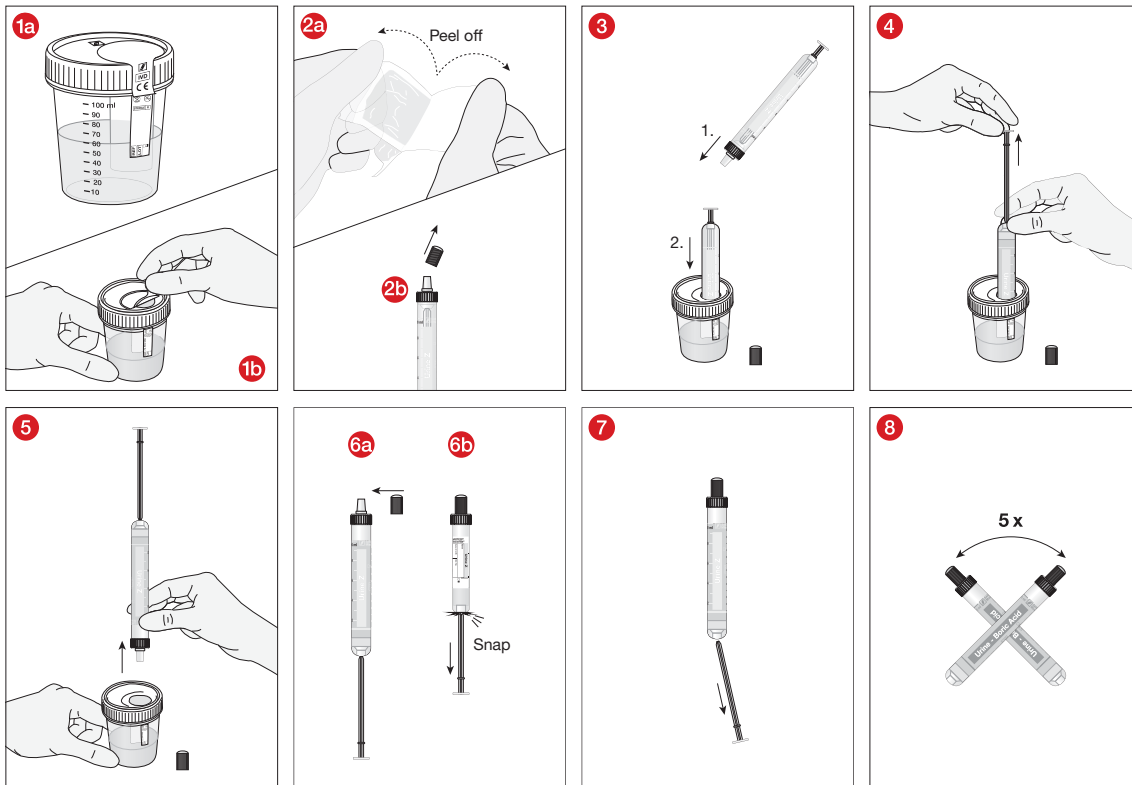


A2: From an NFT urine cup with a fill volume < 20 ml

- 1 Prepare the urine sample for withdrawal by carefully swirling the urine cup, then open it.
- 2 Only for individually sterile-wrapped Urine Monovettes: Open the blister package with the thumb and index finger using the peel-off technique. The transparent film must be facing upwards (Fig. 2a). Hold the Urine Monovette® upright with the tip pointing upwards and remove the stopper on the Luer screw cap. Set it aside for later. Place a suction tip on the Luer connection and make sure the suction tip is firmly attached (Fig. 2b).
- 3 Now dip the suction tip into the urine cup and withdraw the plunger rod until the plunger has reached the base line and the Urine Monovette® is filled with urine. Take the Urine Monovette® out of the liquid and hold it upright with the suction tip pointing upwards. To empty the suction tip, pull the plunger rod downwards until it stops.
- 4 Remove the suction tip and dispose of it in a disposal container for hazardous biological waste.
- 5 Close the Urine Monovette® securely with the previously removed stopper (Fig. 5a). Only when using a Urine Monovette® 3.2 ml: Withdraw the plunger rod until it audibly clicks into the final position (Fig. 5b).
- 6 Break off the plunger rod.
- 7 Only when using a Urine Monovette® Boric acid: Invert the Urine Monovette® Boric acid around 5 times or until the boric acid is completely dissolved.
- 8 Place the Urine Monovette® in a stand.
- 9 Dispose of the NFT urine cup in a container for hazardous biological waste.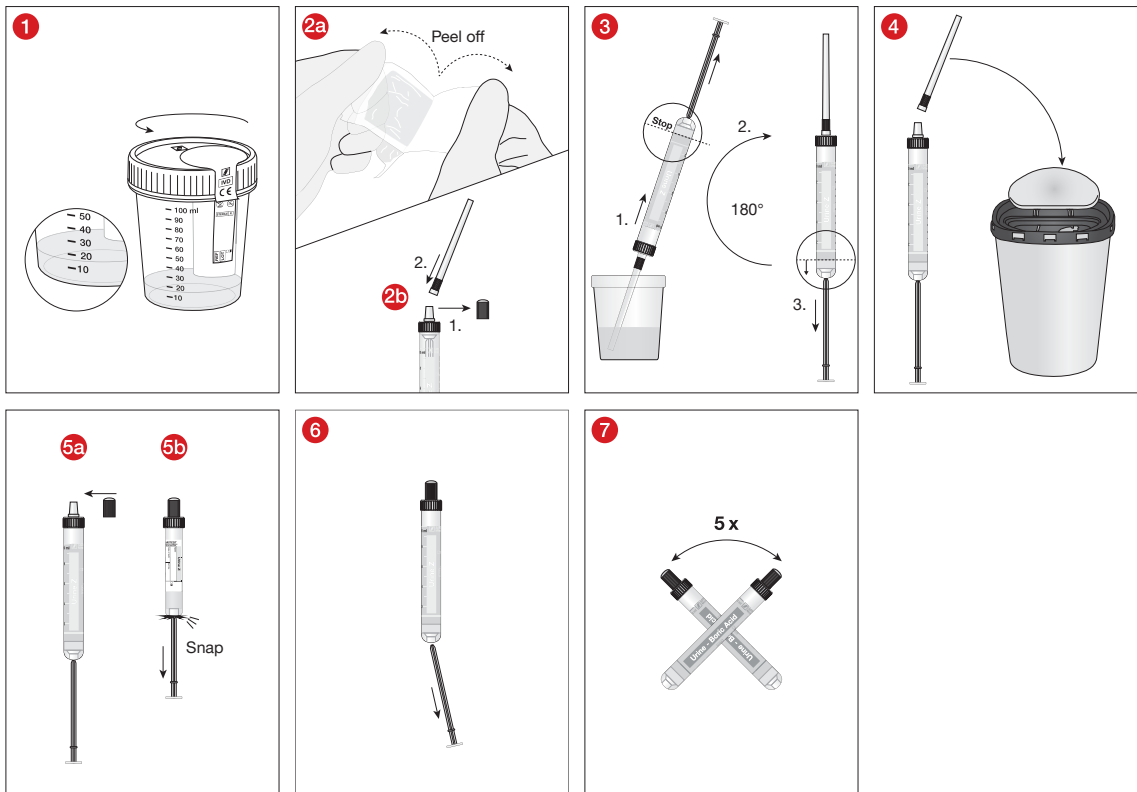


B1: From an NFT urine collection container with fill volume > 1,000 ml

- 1 Prepare the urine specimen for removal by carefully swirling the urine collection container back and forth.
- 2 Set the NFT urine collection container in an upright position.
- 3 Hold the Urine Monovette® upright with the tip pointing upwards and remove the stopper on the Luer screw cap. Set it aside for later.
- 4 First guide the Urine Monovette® with the Luer connector into the needle-free transfer unit and make sure that the Luer cone fully penetrates the membrane and the Urine Monovette® is firmly in place.
- 5 Hold the NFT urine collection container by the recessed grips and bend it forward into a horizontal position, the recessed grips are on top.
- 6 Use the plunger rod to pull the plunger to the bottom of the Urine Monovette® and wait until no more urine flows.
- 7 Set the NFT urine collection container back in an upright position. Detach the Urine Monovette® from the needle-free transfer unit and turn it upright so that the tip points upwards. The NFT urine collection container will automatically seal tightly after removal.
- 8 Close the Urine Monovette® securely with the previously removed stopper (Fig. 8a). Only when using a Urine Monovette® 3.2 ml: Withdraw the plunger rod until it audibly clicks into the final position (Fig. 8b).
- 9 Break off the plunger rod.
- 10 Place the Urine Monovette® in a stand or another suitable device.
- 11 Dispose of the NFT urine collection container in a disposal container for hazardous biological waste.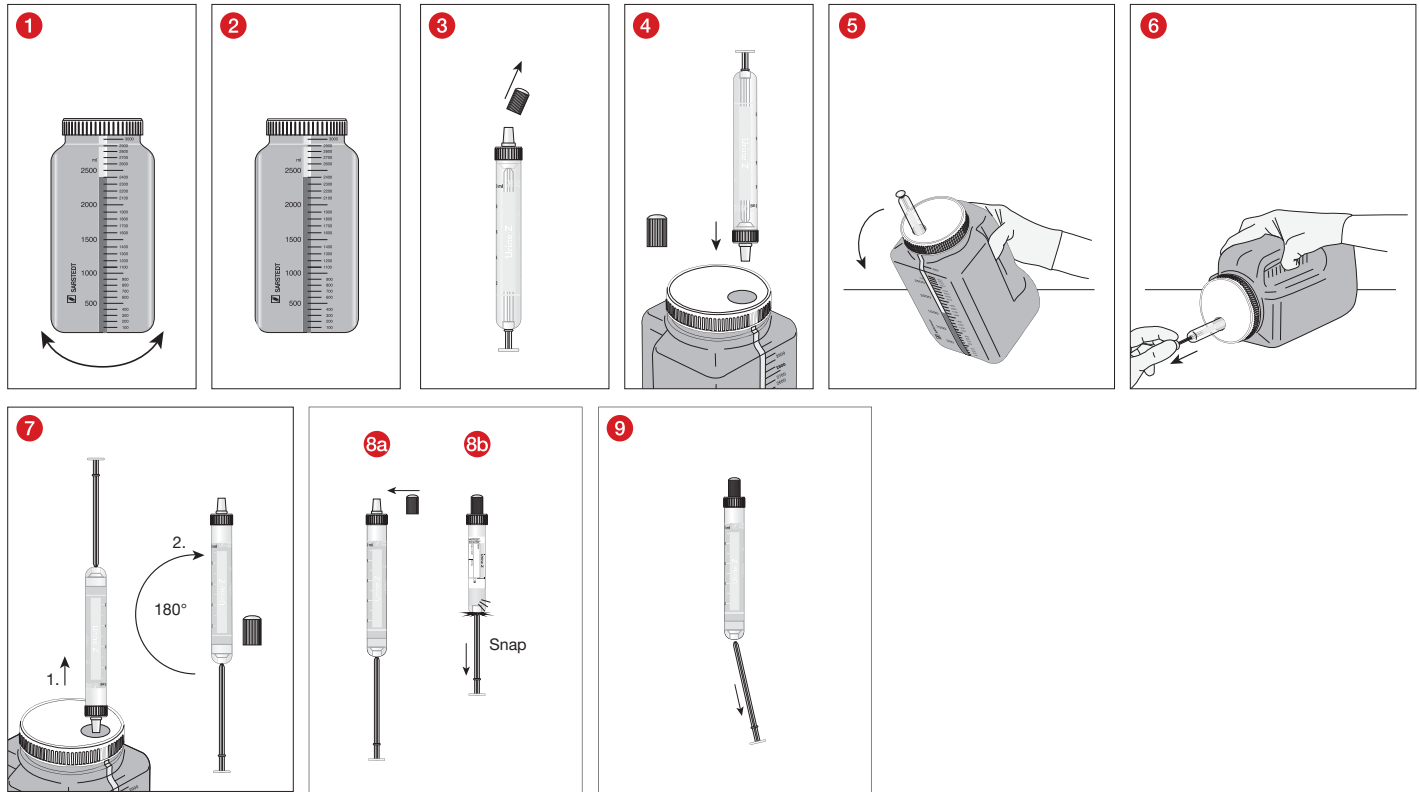


B2: From an NFT urine collection container with transfer unit with fill volume 1,000 – 400 ml

- 1 Prepare the urine specimen for removal by carefully swirling the urine collection container back and forth.
- 2 Set the NFT urine collection container in an upright position.
- 3 Hold the Urine Monovette® upright with the tip pointing upwards and remove the stopper on the Luer screw cap. Set it aside for later.
- 4 First guide the Urine Monovette® with the Luer connector into the needle-free transfer unit and make sure that the Luer cone fully penetrates the membrane and the Urine Monovette® is firmly in place.
- 5 Lift the NFT urine collection container from the work area by the recessed grips and bend it forward so that the opening is tilted downwards slightly. If necessary, the NFT urine collection container can also be placed on an inclined surface.
- 6 Use the plunger rod to pull the plunger to the bottom of the Urine Monovette® and wait until no more urine flows.
- 7 Set the NFT urine collection container back in an upright position on the work area. Detach the Urine Monovette® from the needle-free transfer unit and turn it upright so that the tip points upwards. The NFT urine collection container will automatically seal tightly after removal.
- 8 Close the Urine Monovette® securely with the previously removed (Fig. 8a). Only when using a Urine Monovette® 3.2 ml: Withdraw the plunger rod until it audibly clicks into the final position (Fig. 8b).
- 9 Break off the plunger rod.
- 10 Place the Urine Monovette® in a stand.
- 11 Dispose of the NFT urine collection container in a disposal container for hazardous biological waste.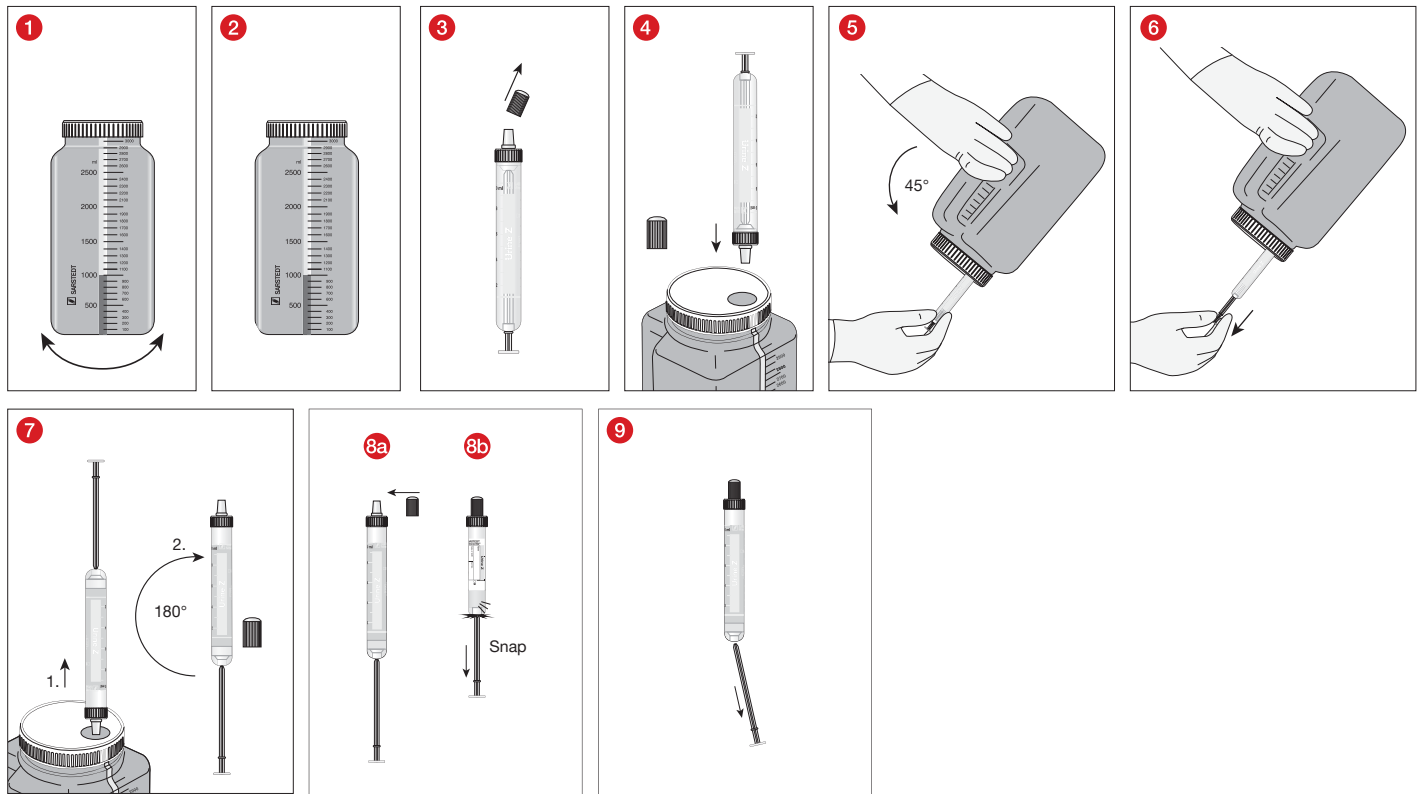

B3: From an NFT urine collection container with fill volume < 400 ml

- 1 Prepare the urine specimen for removal by gently swirling the urine collection container and transfer a sufficient quantity to a urine cup.
- 2 Hold the Urine Monovette® upright with the tip pointing upwards and remove the stopper on the Luer screw cap. Set it aside for later. Place a suction tip on the Luer connection and make sure the suction tip is firmly attached.
- 3 Now dip the suction tip into the urine cup and withdraw the plunger rod until the plunger has reached the base line and the Urine Monovette® is filled with urine. Take the Urine Monovette® out of the liquid and hold it upright with the suction tip pointing upwards. To empty the suction tip, pull the plunger rod downwards until it stops.
- 4 Remove the suction tip and dispose of it in a disposal container for hazardous biological waste.
- 5 Close the Urine Monovette® securely with the previously removed stopper (Fig. 5a). Only when using a Urine Monovette® 3.2 ml: Withdraw the plunger rod until it audibly clicks into the final position (Fig. 5b).
- 6 Break off the plunger rod.
- 7 Place the Urine Monovette® in a stand.
- 8 Close the urine cup and the NFT urine collection container and dispose of both in a container for hazardous biological waste.

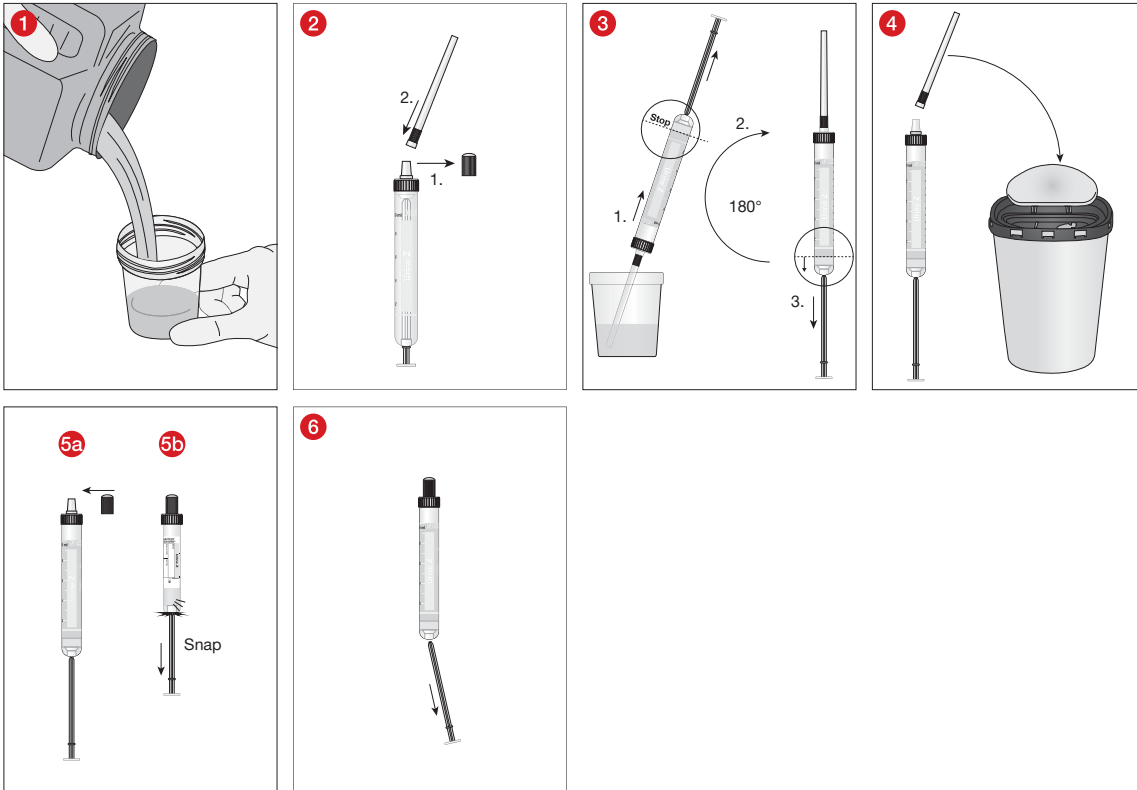Freezing/thawing

THE FOLLOWING GENERAL RECOMMENDATIONS FOR FREEZING APPLY TO URINE MONOVETTES:

Freezing below 0°C

- Check beforehand whether freezing will adversely affect the urine sample or analysis (parameter stability, interference: e.g. haemolysis).
Important: Do not freeze urine samples if they are to be used for urine sediment.
NOTE: For stability of the parameters, consult the instructions for use of the test reagents/analysis device manufacturers.
- The strength values of plastics are reduced in the temperature range below 0°C. Mechanical stress should therefore be avoided.
- The conditions for freezing must be selected so that the contents of the Urine Monovette® freeze evenly or from bottom to top. The Urine Monovette® should have enough room in a holder or storage box to allow it to expand. Holders made of Styrofoam or metal are unsuitable as they can lead to expansion tears.

Freezing at -20°C

- Cool the Urine Monovette® in an upright position over a period of 45–60 min. from room temperature to +4 °C before it can be frozen at -20 °C.

Freezing below -20°C

- Deep-freezing to below -20 °C has not been tested by the manufacturer.
Due to the number of possible influencing factors, it is recommended to conduct freezing tests under routine laboratory conditions.

Thawing

- Allow the Urine Monovette® to thaw for at least 45 min. at room temperature in an upright position. Mechanical stress must also be prevented in this step. Thawing too quickly can impair the results of the analysis.

Centrifugation

IMPORTANT! SARSTEDT Urine Monovettes are designed for maximum 4000 x g (RCF - relative centrifugal force – G-force).

Centrifuge inserts should be selected according to the size of the Urine Monovettes used. The relative centrifugal force has the following ratio to the selected rpm:

$$\text{RCF (g-force)} = 11.2 \times r \times (\text{rpm}/1000)^2$$

"RCF": "Relative centrifugal force"

"rpm": "Revolutions per minute" (min⁻¹)

"r" [in cm]: "Radius of rotation from the middle of the centrifuge to the bottom of the Urine Monovette®."

Use suitable tube holders or inserts only. Centrifuging Urine Monovettes with cracks or centrifuging at an excessive centrifugal force can lead to breakage of the Urine Monovettes, which can release potentially hazardous substances.

Urine Monovettes should be centrifuged in accordance with the centrifugation conditions listed below. If other conditions are used, they must be validated by the user themselves.

It must be ensured that the Urine Monovettes fit well in the centrifuge inserts. Urine Monovettes that protrude above the insert can get caught in the centrifuge head and be broken. Ensure that the centrifuge is filled evenly. Please refer to the centrifuge instructions for use for further information.

CAUTION! Do not remove broken Urine Monovettes by hand.

Refer to the centrifuge instructions for use for information on disinfecting the centrifuge.

Centrifugation recommendation:

The recommended RCF force (g-force) for urine samples for sediment extraction is 400 x g for 5 minutes.

Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. The products must be disposed of in suitable containers for hazardous biological waste that can be subsequently autoclaved and incinerated.
4. Dispose of potentially contaminated material according to the regulations and guidelines of the facility.
5. Please take note of the information in the safety data sheet.

Product-specific standards and guidelines as amended / updated

DIN EN 14254: *In-vitro* diagnostic medical devices - Single-use receptacles for the collection of specimens, other than blood, from humans
 CLSI* GP16 "Urinalysis Approved Guideline"
 *CLSI (Clinical and Laboratory Standards Institute)

Further literature:

European Urinalysis Guidelines: Scand J. Clin. Lab. Invest 2000; 60: 1–96.

Key for symbols and labels:

	Article number
	Batch number
	Use before
	CE marking
	<i>In vitro</i> diagnostic device
	Follow the instructions for use
	If reused: Risk of contamination
	Keep away from sunlight
	Store in a dry place
	Manufacturer
	Country of manufacture
	Danger

The following is also applicable to sterile products:

	Do not use if package is damaged
	Single sterile barrier system with protective packaging outside
	Single sterile barrier system
	Radiation sterilisation
	Do not resterilise

Technical modifications reserved.

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

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