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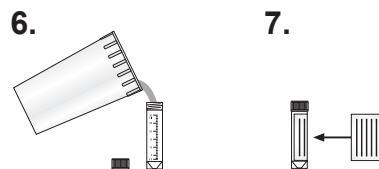
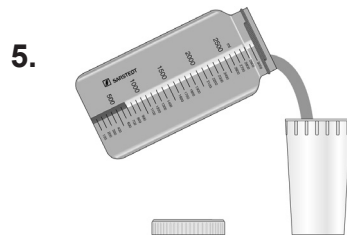
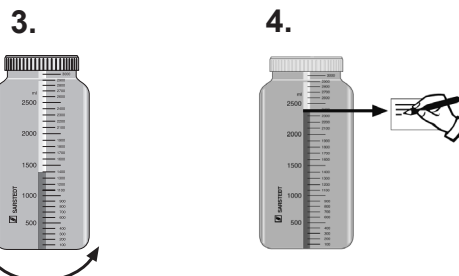
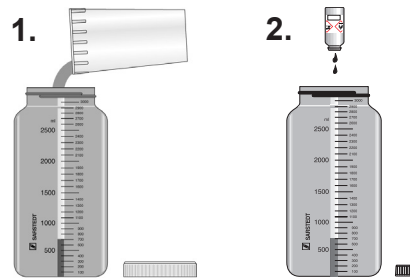
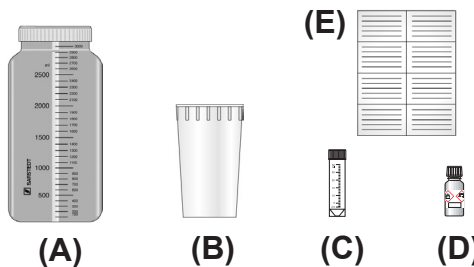
After getting up in the morning empty urinary bladder into the toilet. Write down the date and time on the enclosed label (E).

- Please use the receptacle (B) for the following mictions and fill this urine into the urine container (A).
- Then drip the complete contents (acid) from the small glass bottle (D) into the urine container (A).

Danger  
H 226  
H 290  
H 314

**The small glass bottle contains 9 ml acetic acid 96 %**

Flammable liquid and vapour.  
May be corrosive to metals.  
Causes severe skin burns and eye damage.  
Store locked up and out of the reach of children.  
If swallowed: Rinse mouth. Do not induce vomiting.  
If on skin (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.  
If in eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.  
If exposed or if you feel unwell: Immediately call a poison center/doctor.  
If medical advice is needed, have product container or label at hand.

- Close the urine container (A) well and shake it. Collect any further urine, also in case of bowel movement, and fill it into the urine container (A). Close the urine container (A) well after every use and shake it. The next day, go to the toilet at the time noted down the day before and collect this urine. Collection is completed now.
- Read the collected quantity from the scale on the urine container (A).  
**Write down this figure on the label (E).** Fill in the label (E) completely.
- Fill some urine from the urine container (A) into the receptacle (B).
- Fill the 30 ml tube (C) with urine to the 30 ml mark with urine. Empty the remaining urine from the receptacle (B) and the urine container (A) into the toilet.
- Close the tube (C). Attach the completed label (E) onto the tube (C). Hand the filled tube (C) over to your physician. All components of the urine collection set can be disposed of as domestic waste.

- (A): Urine Container (B): Receptacle (C): 30 ml tube  
(D): Glass bottle contains 9 ml acetic acid 96 % (E): Label for 30 ml tube

Store at room temperature · Technical modifications reserve

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