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
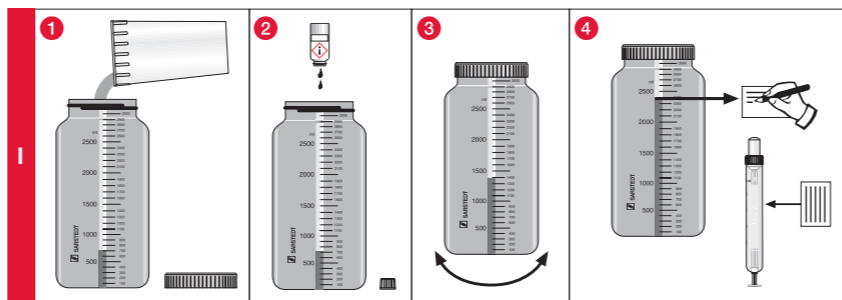
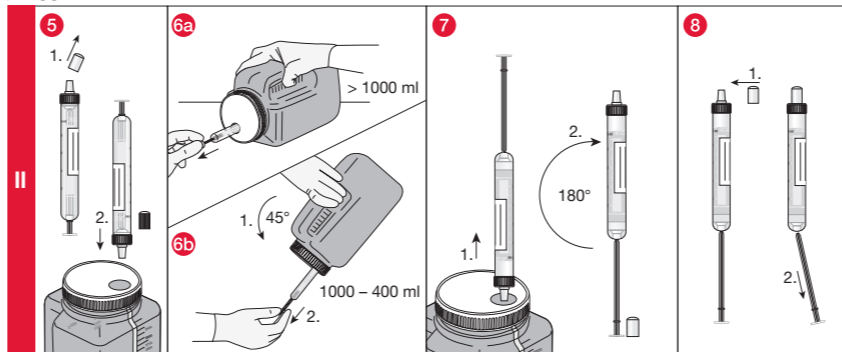
< 400 ml:

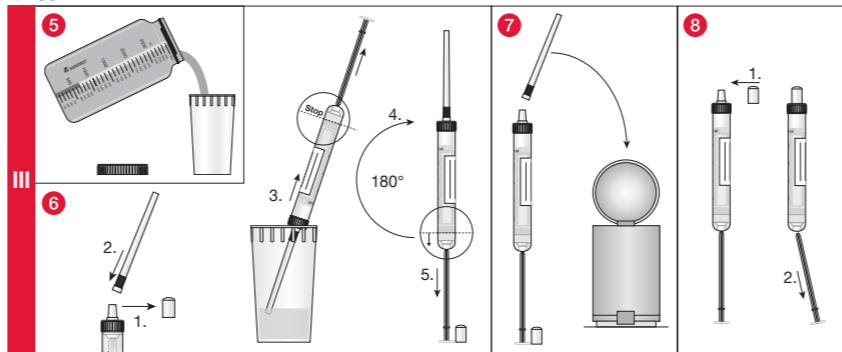EN Instructions for Use UriSet 24 NFT

- After getting up in the morning empty urinary bladder into the toilet. Write down the date and time on the enclosed label (E).
- Please use the receptacle (B) for the following mictions and fill this urine into the urine container NFT (A).
- Then drip the complete contents (acid) from the small glass bottle (D) into the urine container NFT (A).

The small glass (D) bottle contains 9 ml hydrochloric acid 20%

Attention
H315 Causes skin irritation.
H319 Causes serious eye irritation.
H335 May cause respiratory irritation.

- Close the urine container NFT (A) well and shake it. Collect any further urine, also in case of bowel movement, and fill it into the urine container NFT (A). Close the urine container NFT (A) well after every use and shake it. The next day, go to the toilet at the time noted down the day before and collect this urine. Collection is completed now.
- Read the collected quantity from the scale on the urine container NFT (A). **Write down this figure on the label (E).** Fill in the label (E) completely. Attach the completed label (E) to the Urine Monovette® (C).

II > 400 ml **5 1.** Remove the stopper from the Urine Monovette® (C) and keep for later use. **2.** Guide the Urine Monovette® with the Luer connector into the needle-free transfer unit and make sure that the Luer cone fully penetrates the membrane and the Urine Monovette® is firmly in place.

6a > 1000 ml: Hold the NFT urine collection container (A) by the recessed grips and bend it forward into a horizontal position, the recessed grips are on top. Use the plunger rod to pull the plunger to the bottom of the Urine Monovette® (C) and wait until no more urine flows. **6b 400 ml – 1.000 ml:** Lift the NFT urine collection container (A) from the work area by the recessed grips and bend it forward so that the opening is tilted downwards slightly. If necessary, the NFT urine collection container (A) can also be placed on an inclined surface. Use the plunger rod to pull the plunger to the bottom of the Urine Monovette® (C) and wait until no more urine flows. **7** Set the NFT urine collection container (A) back in an upright position on the work area. Detach the Urine Monovette® (C) from the needle-free transfer unit and turn it upright so that the tip points upwards.

III < 400 ml **5** Fill some urine from the urine container NFT (A) into the receptacle (B). **6 1.** Remove the stopper from the Urine Monovette® (C) and keep for later use. **2.** Attach the suction tip to the Urine Monovette® (C). **3.** Insert the suction tip into the receptacle (B) and fill the Urine Monovette® (C) up to the base line. **4.** To empty the suction tip, hold the Urine Monovette® (C) in a vertical position, away from the receptacle (B). **5.** Withdraw the piston rod in a downward direction until it locks into its base. **7** Remove the suction tip.

8 Close the Urine Monovette® (C) securely with the previously removed stopper. Break off the plunger rod. Hand the filled tube (C) over to your physician. All components of the urine collection set can be disposed of as domestic waste.

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