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
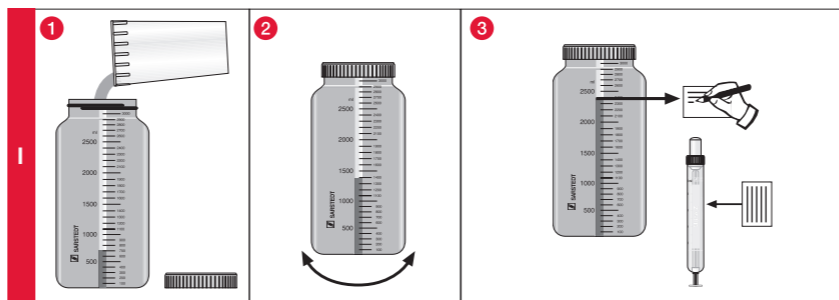
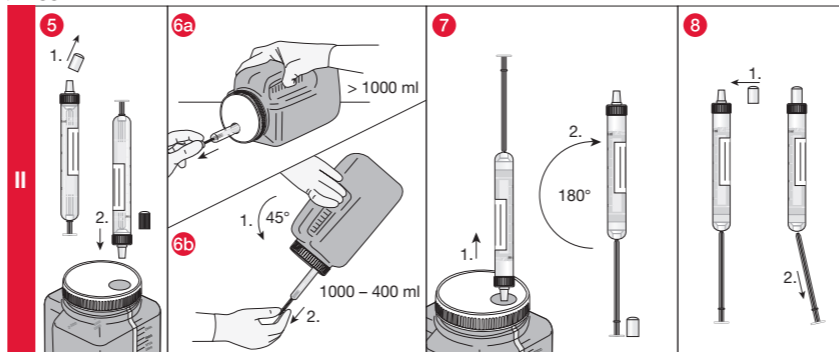
< 400 ml:

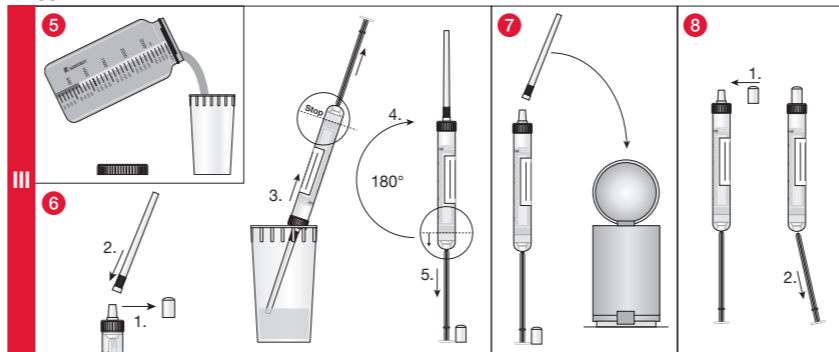**EN Instructions for Use****UriSet 24 NFT**

- I** After getting up in the morning empty urinary bladder into the toilet. Write down the date and time on the enclosed label **(D)**.
- 1** Please use the receptacle **(B)** for the following mictions and fill this urine into the urine container NFT **(A)**.
- 2** Close the urine container NFT **(A)** well. Collect any further urine, also in case of bowel movement, and fill it into the urine container NFT **(A)**. Close the urine container NFT **(A)** well after every use. The next day, go to the toilet at the time noted down the day before and collect this urine. Collection is completed now.
- 3** Read the collected quantity from the scale on the urine container NFT **(A)**. **Write down this figure on the label (D)**. Fill in the label **(D)** completely. Attach the completed label **(D)** to the Urine Monovette® **(C)**.
- 4** Shake the closed urine container several times for optimal mixing of the urine.
- II > 400 ml** **5** **1.** Remove the stopper from the Urine Monovette® **(C)** and keep for later use. **2.** Guide the Urine Monovette® with the Luer connector into the needle-free transfer unit and make sure that the Luer cone fully penetrates the membrane and the Urine Monovette® is firmly in place.
- 6a > 1000 ml:** Hold the NFT urine collection container **(A)** by the recessed grips and bend it forward into a horizontal position, the recessed grips are on top. Use the plunger rod to pull the plunger to the bottom of the Urine Monovette® **(C)** and wait until no more urine flows.
- 6b 400 ml – 1.000 ml:** Lift the NFT urine collection container **(A)** from the work area by the recessed grips and bend it forward so that the opening is tilted downwards slightly. If necessary, the NFT urine collection container **(A)** can also be placed on an inclined surface. Use the plunger rod to pull the plunger to the bottom of the Urine Monovette® **(C)** and wait until no more urine flows. **7** Set the NFT urine collection container **(A)** back in an upright position on the work area. Detach the Urine Monovette® **(C)** from the needle-free transfer unit and turn it upright so that the tip points upwards.
- III < 400 ml** **5** Fill some urine from the urine container NFT **(A)** into the receptacle **(B)**. **6** **1.** Remove the stopper from the Urine Monovette® **(C)** and keep for later use. **2.** Attach the suction tip to the Urine Monovette® **(C)**. **3.** Insert the suction tip into the receptacle **(B)** and fill the Urine Monovette® **(C)** up to the base line. **4.** To empty the suction tip, hold the Urine Monovette® **(C)** in a vertical position, away from the receptacle **(B)**. **5.** Withdraw the piston rod in a downward direction until it locks into its base. **7** Remove the suction tip.
- 8** Close the Urine Monovette® **(C)** securely with the previously removed stopper. Break off the plunger rod. Hand the filled tube **(C)** over to your physician. All components of the urine collection set can be disposed of as domestic waste.

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